GONIO ready®

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Instruction Manual

Scope of Delivery

Components	Order Number
GONIO ready® 200 for f = 200 mm (box of 6 units)	10008420
GONIO ready® 175 for f = 175 mm (box of 6 units)	10011226
Including	
<ul style="list-style-type: none"> ■ Sterility disc for adapter plate 	
Instruction Manual GONIO ready®	G/10011226/.../en


Scope of delivery subject to change due to ongoing technical development.

- If you discover transport damage upon delivery, please report this to the shipping company immediately.

Safety Instructions

- Carefully study this instruction manual.
- Follow all safety instructions in this manual.
- Follow the safety instructions for the surgical microscope and any other equipment used in the procedure.
- The GONIO ready® is only intended for use by trained ophthalmic surgeons and their OR staff.
- Familiarize yourself with the operation of the device prior to use. Consult OCULUS Surgical or authorized dealer with any questions.
- Verify the integrity of the packaging and device before use. Pay attention to the storage conditions; see page 16.
- Do not install or remove the GONIO ready® over the patient.
- Comply with all applicable local laws and follow your facility's hygiene and disposal regulations.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer (vigilance@oculus.de) and the competent authority of the region in which the user and/or patient is established.

Device Components

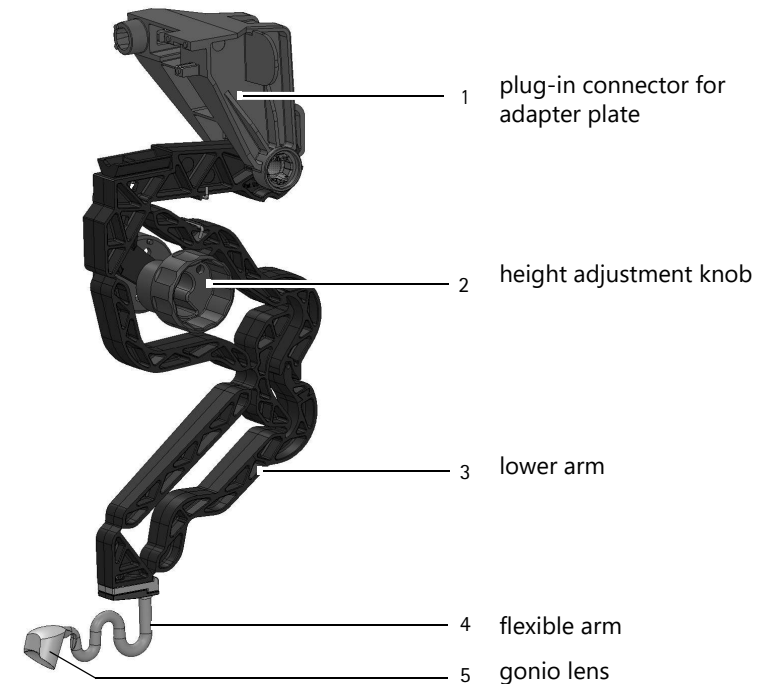Accessories

Sterility discs for adapter plate

Intended Use / Indications for use

This Gonioscopic Viewing System (GONIO ready®) is intended for use in conjunction with compatible surgical microscopes in hospitals, clinics and other medical institutions.

The GONIO ready® is indicated for viewing ocular structures of the Anterior Chamber. There are no known contraindications, undesirable side effects, or specific patient population.

The GONIO ready® is intended for single use only. It may only be used in a sterile condition. It cannot be re-processed for reuse.

The GONIO ready® is intended for use with a microscope objective focal length of $f = 175 \text{ mm}$ or $f = 200 \text{ mm}$.

The surgical microscopes must be designated as adaptable by OCULUS Optikgeräte GmbH; see page 4.

Mode of Operation

The GONIO ready® is a gonioscopic viewing system which attaches to an ophthalmic operating microscope and positions a direct gonioscopy lens on the patient's cornea. The GONIO ready® enables hands-free observation of the anterior chamber angle and true bi-manual operation during Minimally Invasive Glaucoma Surgery (MIGS).

The combination of a surgical microscope and the direct gonioscopy lens of the GONIO ready® provides a stereoscopic and non-inverted view of the anterior chamber.

The GONIO ready® requires tilting of the microscope 30 ± 15 degrees towards and/or positioning of the patient's head 15 ± 10 degrees away to achieve desired visualization of the angle. A combined viewing angle of 45 degrees off-axis is recommended.

An ophthalmic lubricant/viscoelastic, such as ImproVue® Ophthalmic Lubricant Drops (where available), must be applied as a coupling medium between the gonio lens and cornea before use.

The GONIO ready® image is focused using the microscope focus adjustment which also adjusts the height of the gonio lens. To keep the gonio lens in proper contact with the cornea, the height adjustment knob on the GONIO ready® may be adjusted after focusing; see page 15.

Microscope Adaptation


Caution

Risk to the patient if microscope adapter plate is installed incorrectly

- Do not install microscope adapter plate over patient
- Ensure thumbscrew is tightened firmly and adapter plate is engaged securely

Prior to installation of GONIO ready®, perform installation of unsterile microscope adapter as follows:

- ➔ Slide adapter plate into mating dovetail on bottom of microscope until engaged completely and cannot advance further.
- ➔ Tighten thumbscrew to secure adapter within dovetail.

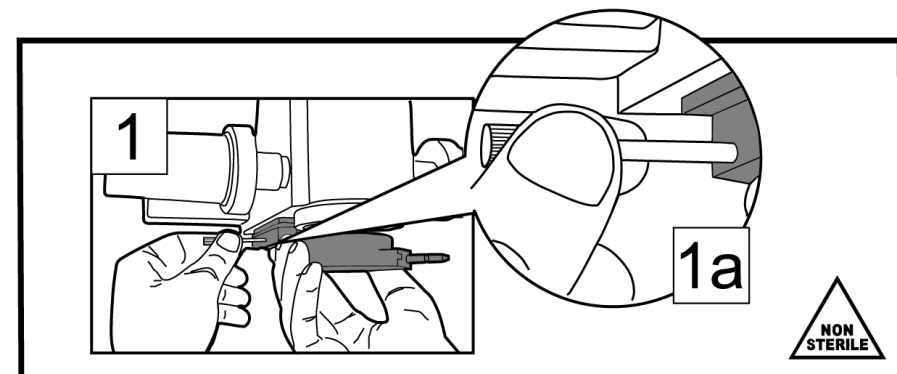Unpacking

Caution

Risk to the patient: Contamination if unpacked incorrectly
GONIO ready® is delivered in sterile condition. Incorrect opening or removal may compromise sterility and render device unusable.

- When unpacking, ensure that all components of GONIO ready® remain sterile.
- Before use, verify storage conditions; see page 16.

GONIO ready® is double-packed

- Non-sterile outer packaging
- Sterile inner packaging

Unpacking Instructions

Non-sterile personnel:

- Check outer packaging for damage. DO NOT USE if packaging is damaged.
- Open outer packaging without touching inner packaging.
- Make sure that sterile components of GONIO ready® do not become contaminated and thus unsterile.

Sterile personnel:

- Remove blister containing GONIO ready® from outer packaging, and place on sterile field.
- Carefully unsnap and remove protective clamshell around lens, taking care to not damage lens or flexible arm.


Installation Instructions


Caution

Risk to the patient if GONIO ready® is installed incorrectly


- Do not install GONIO ready® over the patient.
- Follow appropriate aseptic techniques during device installation.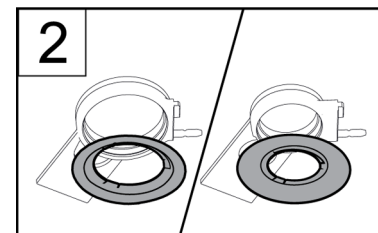

- Select appropriate size sterility disc for microscope adapter.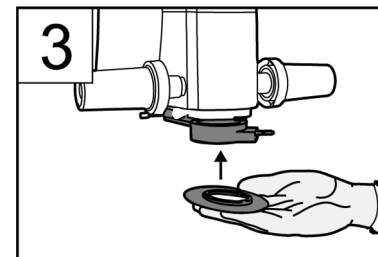

- Without touching unsterile adapter plate or microscope, snap sterility disc with side labeled "TOP" facing up, into adapter plate until sterility disc sits flush and engages fully (fits GONIO ready® adapter plate only).

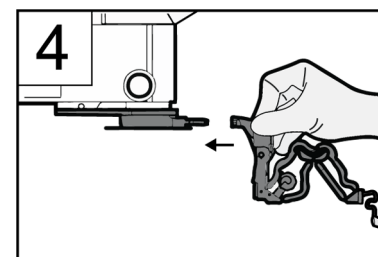- Attach GONIO ready® device onto microscope adapter plate until a click is heard and plug-in connector is fully engaged.

Instructions for Use

Caution

Risk of eye injury as a result of excessive changes of the working distance between GONIO ready® and the patient.

When GONIO ready® is in the working position and/or gonio lens is in contact with eye, the following instructions must be followed:

- Do not use XYZ reset function on the microscope.
- Do not adjust height of the stand arm, either by motor or manually, over surgical area.
- Do not change the patient's position by adjusting height of OR-table.

Caution




Risk to the patient as a result of uncontrolled patient head movement.

- Take suitable measures to prevent uncontrolled head movements of the patient during surgical procedure.

- Use standard microscope settings in accordance with manufacturer's specifications.
- In an extreme emergency, e.g. uncontrolled downward movement of microscope:
 - Stop by pressing Emergency Off button.
 - Pull / move entire surgical microscope upwards.
 - Swing GONIO ready® out of microscope beam path.
- If top surface of lens needs cleaning: Swing GONIO ready® out of microscope beam path to standby position, or lift lower arm of GONIO ready® to facilitate cleaning of lens with sterile swab. Clean gonio lens with care, as lens can easily be damaged.

Focusing Instructions

To obtain best focus of the iridocorneal angle, heed the following steps: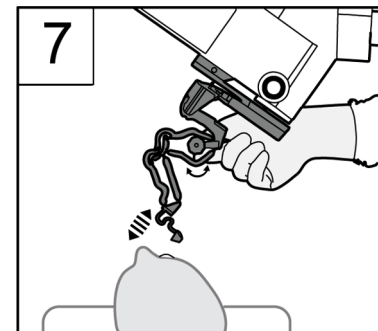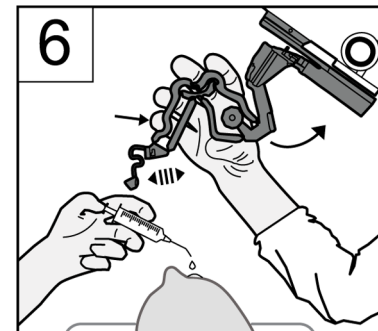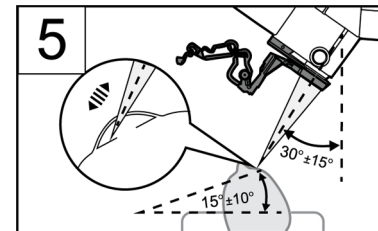

- After performing surgical steps needed prior to gonioscopy, establish desired tilt of microscope head; $30^\circ \pm 15^\circ$ is recommended (Fig 5).
- Additional tilting of the patient's head will provide the best view of the iridocorneal angle; $15^\circ \pm 10^\circ$ is recommended (Fig 5).
- With a total viewing angle of appx. 45° established, center the nasal angle within microscope view.
- Obtain best focus on the nasal iris or inferior/superior limbal vasculature under high magnification.
- Reduce magnification to lowest setting.
- Apply ophthalmic lubricant (Impr-oVue® Ophthalmic Lubricant Drops, where available) or a Dispersive OVD onto the cornea (Fig 6). This ensures optimal optical coupling and image quality.
- Lift lower arm of GONIO ready® with a finger and swing GONIO ready® to working position as shown (Fig 6).
- Center lens over the cornea using X-Y control on microscope foot-pedal.
- Carefully lower gonio lens into contact with the patient's eye by using blue height adjustment knob (Fig 7).

- Take special care to monitor position of gonio lens throughout procedure.
- Make slight adjustments with microscope focus to optimize view as needed.


Adjusting the view

Caution

Risk to the patient:

- When focusing microscope down, do not exceed the free motion range of flexible lower arm of GONIO ready®.

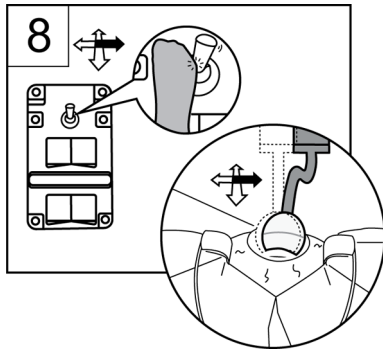- Further positioning adjustments should be made with X-Y control on microscope foot-pedal (Fig 8).
- Adjusting focus in the upward direction may result in separation of gonio lens from cornea and obscure view of the iridocorneal angle. If this should occur, use blue height adjustment knob to lower lens and re-establish contact with the cornea.
- Pressure of lens on cornea can be modified with blue height adjustment knob. Excessive pressure may destabilize lens on the eye and/or impede free movement of lens.

→ If desired, lens can be rotated to expand field of view by manually swiveling black lower arm about the adapter.

Removal and Disposal after Surgery

After surgery, remove and dispose of GONIO ready® in accordance with applicable hygiene regulations.

Caution

Risk to the patient if GONIO ready® is removed incorrectly

- Do not remove GONIO ready® over the patient.

- Pull GONIO ready® off adapter plate.
- Remove sterility disc from adapter plate.
- Dispose of parts in accordance with your facility's contaminated medical waste guidelines.

Storage

→ Heed the following information:

Type of packaging	Storage
Cardboard packaging	<ul style="list-style-type: none"> ■ Unopened or sealed dust-tight ■ In a dark room ■ Ambient conditions; see page 3 ■ Storage duration: See expiration date on packaging
Non-sterile outer packaging	<ul style="list-style-type: none"> ■ Protected storage (e.g. in cabinets or drawers): 6 Months, but no later than expiration date ■ Unprotected storage (e.g. on shelves): Use within 48 hours
Sterile inner packaging	<ul style="list-style-type: none"> ■ For immediate use

Troubleshooting

Caution

Risk to the patient if GONIO ready® is defective

- If a fault occurs that you cannot rectify by following the instructions below, dispose of GONIO ready® in accordance with your facility's contaminated medical waste guidelines

Fault	Possible Cause	Remedy
Adapter or dovetail mount is wobbling	<ul style="list-style-type: none"> ■ Knurled screw is loose ■ Screws are loose 	<ul style="list-style-type: none"> ■ Ensure adapter plate is fully engaged and tighten knurled screw ■ Tighten screws with a suitable screwdriver
Cropped or decentered image	<ul style="list-style-type: none"> ■ GONIO ready®, other components, or GONIO ready® adapter have been mounted skewed 	<ul style="list-style-type: none"> ■ Check that adapter and device are correctly mounted ■ Recenter lens using microscope X/Y
Hazy image	<ul style="list-style-type: none"> ■ Too much ophthalmic lubricant/viscoelastic applied on cornea ■ Insufficient application of ophthalmic solution ■ Optical surfaces are dirty or damaged ■ Microscope is not properly focused 	<ul style="list-style-type: none"> ■ Remove/reapply ophthalmic lubricant/viscoelastic, such as Im-proVue® Ophthalmic Lubricant Drops, onto cornea ■ Inspect and clean optical surfaces ■ Focus microscope in accordance with instructions
Blurred image	<ul style="list-style-type: none"> ■ GONIO ready® model is not compatible with microscope objective lens ■ Microscope is not properly focused 	<ul style="list-style-type: none"> ■ Check that GONIO ready® model corresponds with microscope objective ■ Focus microscope in accordance with instructions
Insufficient angle observation	<ul style="list-style-type: none"> ■ Gonio lens is incorrectly positioned ■ Microscope head is not tilted ■ Lens is not making good contact with viscoelastic solution ■ Excessive pressure on cornea 	<ul style="list-style-type: none"> ■ Carefully adjust lens position with microscope X-Y control ■ Tilt microscope head to recommended position ■ Carefully lower gonio lens with height adjustment knob ■ Carefully raise gonio lens with height adjustment knob
Strong reflections on patient's eye or lens	<ul style="list-style-type: none"> ■ Microscope illumination is too high 	<ul style="list-style-type: none"> ■ Reduce illumination intensity in microscope settings
Lens does not remain in contact with patient's eye	<ul style="list-style-type: none"> ■ Edge of lens is too close to surgical insertion/limbus ■ Lens is not centered on eye ■ Lens is positioned too high 	<ul style="list-style-type: none"> ■ Recenter lens using microscope X/Y ■ Adjust vertical position of lens using blue height adjustment knob

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OCULUS Surgical Inc.