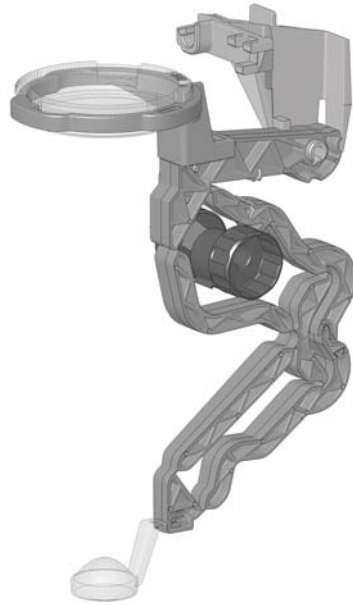
BIOM[®] ready

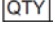
für den Einmalgebrauch, für f=175 mm oder f=200 mm
 disposable, for f=175 mm or f=200 mm
 desechable para un solo uso, para f=175 mm ó f=200 mm
 jetable, pour f=175 mm ou f=200 mm
 monouso, per f=175 mm o f=200 mm
 para uma utilização, para f=175 mm ou f=200 mm
 Для одноразового пользования, для f=175 мм или f=200 мм

Gebrauchsanweisung de, Seite 5
Instruction Manual en, page 13
Manual de Instrucciones es, página 21
Mode d'emploi fr, page 29
Instruzioni per l'uso it, pagina 37
Manual de instruções pt, página 45
Руководство по эксплуатации ru, стр. 53

Symbole / Symbols / Símbolos / Symboles / Simboli / Símbolos / Символы

	Hersteller Manufacturer Fabricante Fabricant Produttore Fabricante Производитель		Herstellungsdatum Manufacturing date Fecha de fabricación Date de fabrication Data di produzione Data de fabricaço Дата производства
	Nicht mit einem Messer öffnen Do not open with a knife No utilice cuchillo para abrir Ne pas ouvrir avec un couteau Non aprire con il coltello Não abra com uma faca! Не открывать ножом		Verwendbar bis Use by Fecha de caducidad À utiliser avant Utilizzabile fino al Utilizável até ИСПОЛЬЗОВАТЬ до
	Bei beschädigter Verpackung nicht verwenden Do not use when packaging is damaged No utilizar si el embalaje está dañado Ne pas utiliser si l'emballage est endommagé Non utilizzarlo se la confezione è danneggiata Com embalagem danificada não utilizar Не использовать в случае повреждения упаковки		Temperaturbegrenzung Lagerung Storage temperature limit Temperaturas limite de almacenamiento Températures de stockage admissibles Limite di temperatura per la tenuta in magazzino Limite de temperatura armazenamento Ограничение температуры хранения

	Relative Luftfeuchte / relative air humidity / humidité relative/ umidità atmosferica relativa / humedad relativa humidade do ar relativo относительная влажность		Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
	Trocken aufbewahren Keep in a dry place Conservar seco Conserver au sec Luogo asciutto Guardar em lugar seco Сухое место / сухо сохраняют		einmal verwenden Use once De un solo uso. Utiliser une seule fois. Monouso Usar uma vez Использовать один раз
	steriles Produkt, einmal verwenden Sterile product, use once Producto estéril de un solo uso produit stérilisé, à n'utiliser qu'une seule fois Prodotto sterile, usa e getta Produto esterilizado, usar uma vez Стерильное изделие одноразового применения		CE Kennzeichnung mit benannter Stelle CE marking with notified body Cód. de pedido (cód. de artículo) Marcado CE con organismo notificado Marquage CE avec identification de l'organisme notifié Marchio CE con l'organismo notificato Marcação CE com organismo notificado Маркировка CE с указанием уполномоченного органа
	Stückzahl Quantity Nº de unidades Quantité N. di pezzi... N.º de peças Количество		Sterilisiert mit Ethylenoxid Sterilized with ethylene oxide Esterilizado con óxido de etileno Stérilisé avec de l'oxyde d'éthylène Sterilizzato con ossido di etilene Esterilizado com óxido de etileno Стерилизация этиленоксидом

	Gebrauchsanweisung beachten Observe instruction sheet Siga las instrucciones de uso Observer la notice d'utilisation Osservare le Istruzioni per l'uso Observar as instruções de utilização Следуйте инструкции по эксплуатации		Achtung Attention Atención Attention Attenzione Atenção Внимание
	Enthält kein Naturkautschuklatex Does not contain natural rubber latex No contiene látex de caucho natural Ne contient pas de latex de caoutchouc naturel Non contiene lattice di gomma naturale Não contém látex de borracha natural Не содержит латекс из натурального каучука		Bestellnummer (Artikelnummer) Order number (article number) Cód. de pedido (cód. de artículo) Nº de commande (nº d'article) Numero di ordinazione (numero dell'articolo) Número de encomenda (número do artigo) Номер заказа (номер артикула)
	Losnummer/Chargennummer Lot number/batch number Lote nº Nº du lot/nº de la charge N. lotto/N. partita Número do lote/número da carga Номер партии/Серийный номер		USA: verschreibungspflichtig USA: Only available by prescription EE. UU.: con receta médica USA : uniquement sur ordonnance USA: da vendersi dietro presentazione di ricetta medica USA: receita obrigatória США: отпускается по рецепту

**Technische Daten / Technical data / Datos técnicos /
Caractéristiques techniques / Dati tecnici / Dados técnicos / Технические
характеристики**

Breite / Width / Anchura / Largeur / Larghezza / Largura / Ширина	58 mm (2,27 in)
Tiefe / Depth / Profundidad / Profondeur / Profondità / Profundidade / Глубина	116 mm (4,55 in)
Höhe, maximal / Height, maximum / Altura máxima / Hauteur, maximale / Itezza massima / Altura, máxima / Высота, макс. ■ f = 200 mm ■ f = 175 mm	180 mm (7,1 in) 160 mm (6,31 in)
Hub Sicherheitseinschub, minimal / Travel of safety rod, minimum / Carrera mínima innesso di sicurezza / Déplacement de sécurité, minimal / Corsa minima innesto di sicurezza / Inserção de segurança curso, mínimo / Ход предохранительного выдвигного шасси, мин.	~ 27 mm (1,06 in)
Verstellbereich / Adjustment range / Rango de ajuste / Domaine de réglage / Campo di regolazione / Área de ajuste / Диапазон регулировки	23 mm (0,92 in)
Gewicht / Weight / Peso / Poids / Peso / Peso / Вес ■ f = 200 mm ■ f = 175 mm	~ 50,5 g (1,2 lbs) 47 g (1,03 lbs)

**Umgebungsbedingungen/ Ambient conditions / Condiciones ambientales
/ Conditions ambiantes / Condizioni ambientali / Condições do ambiente
/ Условия окружающей среды**

	Temperatur / temperature / temperatura / température / temperatura / temperatura / температура	Relative Luftfeuchte / relative air humidity / humidité relative / umidità atmosferica relativa / humedad relativa humidade do ar relativo относительная влажность	Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
Transport / transport / transport / trasporto / transporte / transporte / перевоз	-10°C – +50°C (-14°F – +122°F)	10 % – 85 %	700 hPa – 1060 hPa
Lagerung / storage / stockage / stoccaggio / almacenamiento / armazenagem / хранение	+5°C – +50°C (41°F – +122°F)	10 % – 95 %	700 hPa – 1060 hPa
Gebrauch / use / usage / uso / uso / uso употребление	+10°C – +40°C (+50°F – +104°F)	30 % – 90 %	800 hPa – 1060 hPa

**Mikroskope/ Microscopes / Microscopios / Microscope /Microscopi /
Microscópios / Микроскоп**

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Instruction Manual

Scope of Delivery

Components	Order Number
BIOM® ready for f=175 mm	54413
BIOM® ready for f=200 mm	54414
Including	
<ul style="list-style-type: none"> ■ Sterile cover for the adapter plate BIOM® ready ■ Sterile cap for the knurled screw at the dovetail mount ■ Sterile cap for the knob of the SDI® 	
Instruction Manual BIOM® ready	G/54413/.../en

Scope of delivery subject to change due to ongoing technical development.

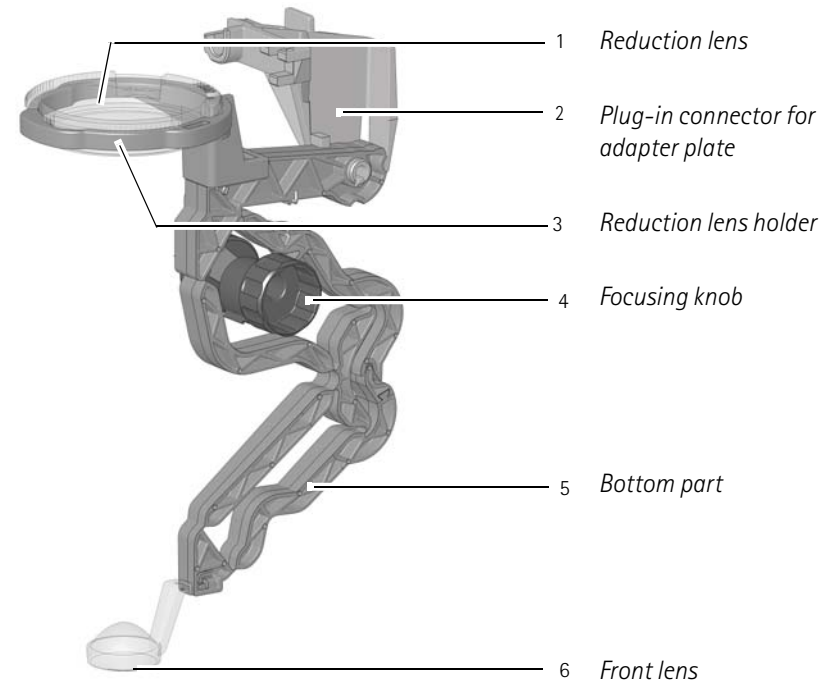
- ➔ If you discover transport damage upon delivery, please report this to the shipping company immediately.

Safety Instructions


- ➔ Read through the Instruction Manual carefully.
- ➔ Heed the following safety instructions.
- ➔ Keep the Instruction manual in a safe place in the vicinity of the unit.
- ➔ Make sure that the BIOM® ready is only used by trained physicians and OR employees, who, due to their knowledge, training and practical experience, can guarantee proper use.
- ➔ Heed the instruction manuals and safety instructions for the surgical microscope and any other equipment you are using.
- ➔ Before use: Get OCULUS or an authorized dealer to train you in the operation of the BIOM® ready.
- ➔ Make sure that the packaging and the BIOM® ready are undamaged and that the BIOM® ready is sterile.
- ➔ Before use, pay attention to the storage conditions; see page 19.
- ➔ Do not mount and dismount the BIOM® ready over the patient.

- ➔ Comply with the legal regulations in force in your country and with the hygiene and disposal regulations of the hospital or the clinic.

Device Components

Accessories

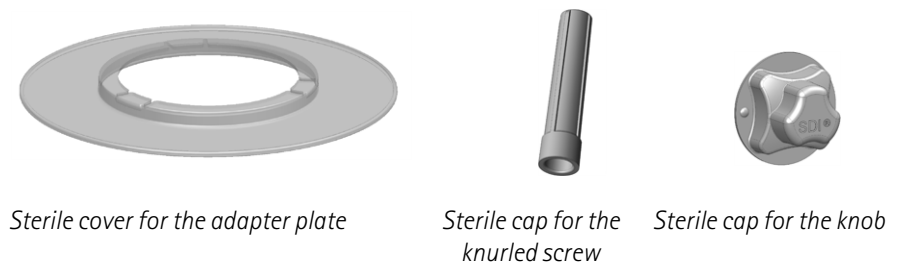Proper Use

This Binocular Indirect Ophthalmo-Microscope (BIOM® ready) is used for non-contact observation of surgical procedures in the posterior section of the eye.

The BIOM® ready is intended for single use only. It may only be used in sterile conditions. It cannot be re-processed for reuse.

The BIOM® ready is intended for use in conjunction with compatible surgical microscopes in hospitals, clinics and other medical institutions.

The BIOM® ready is available in models for a focal length of $f=175$ mm or for a focal length of $f=200$ mm. It has a wide-angle front lens and a reduction lens.

The maximum observation of the fundus is approx. 120° , the working distance (front lens - cornea) is approx. 3-4 mm.

The surgical microscopes must be designated as adaptable by OCULUS Optikgeräte GmbH, see page 4.

Mode of Operation

In conjunction with an SDI (Stereoscopic Diagonal Inverter) for erecting the image, the BIOM® ready enables non-contact, wide-angle observation of the fundus and vitreous cavity.

The combination of a surgical microscope and the optical components of the BIOM® ready allows the vitreous cavity to be observed under stereoscopic conditions. As an indirect ophthalmo-microscope, the BIOM® ready works without touching the cornea during the surgical procedure.

The patient's eyeball can be moved freely during the procedure. Peripheral fundus sections are thus easy to see.

The optical system of the BIOM® ready consists of a reduction lens and a front lens. The reduction lens ensures a more or less constant distance between the patient's eye and the surgical microscope when swinging the BIOM® ready in and out.

The position of the reduction lens with respect to the surgical microscope is fixed.

The BIOM® ready image is focused by adjusting the height of the front lens. The bottom part is used to adjust the height of the front lens. The distance between the surgical microscope and the front lens is set manually using the focusing knob at the BIOM® ready.

Unpacking

Caution

Risk to the patient: Contamination if unpacked incorrectly. The BIOM® ready is sterile-packed. If you unpack the BIOM® ready incorrectly, it can become unsterile and thus unusable.

- When unpacking the BIOM® ready, make sure that it stays sterile.
- Before use, pay attention to the storage conditions; see page 19.

The BIOM® ready is double-packed

- Non-sterile outer packaging
- Sterile inner packaging
- Please heed the following instructions.

Non-sterile personnel:

- Check the outer packaging for damage. If the packaging is damaged, you must not use the BIOM® ready.
- Check whether the focal length of your microscope lens matches the BIOM® ready model.
- Remove the outer packaging. Take care **not** to touch the inner packaging as you do so.
- Make sure that the sterile BIOM® ready does not become contaminated and thus unsterile.

Sterile personnel:

- Remove the BIOM® ready from its sterile packaging.

Mounting Information

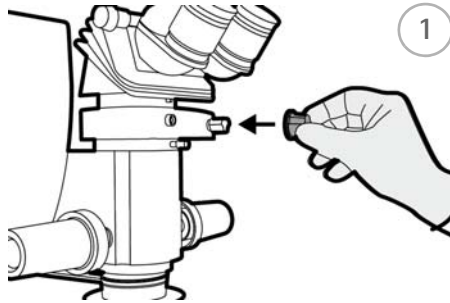
Caution


Risk to the patient if the BIOM® ready is mounted incorrectly.

- Do not mount the BIOM® ready over the patient.


Mounting on a Non-Sterile Adapter Plate

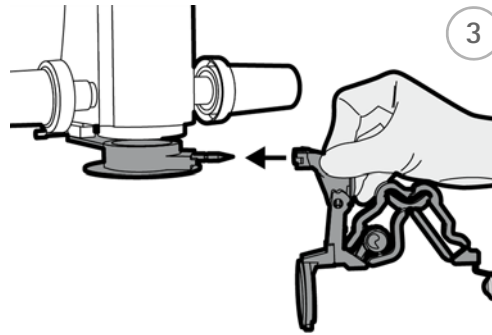
→ Put the sterile cap into place at the SDI®.

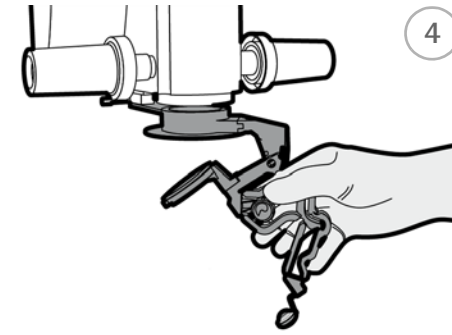→ Using a minimum of force, plug the BIOM® ready onto the adapter plate until it reaches the stop. You must overcome a detent which locks the connection.

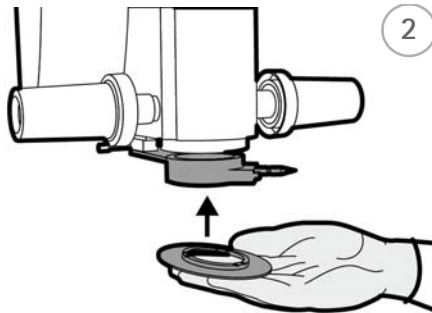→ To swing in the BIOM® ready, lift the bottom part a little.

→ Put the sterile cover over the adapter plate.
(fits adapter plate BIOM® 5/BIOM® ready only).

Caution

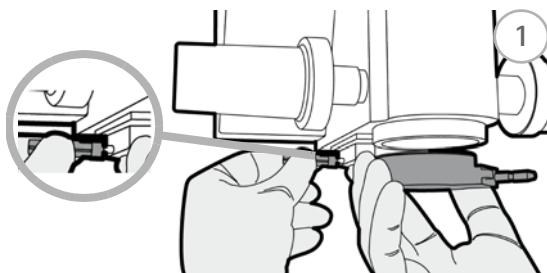
Risk to the patient if the BIOM® ready is inserted incorrectly.


If you do not insert the BIOM® ready correctly, you cannot use it safely and you will get a decentered fundus image.

→ Make sure that the BIOM® ready has been inserted up to the limit stop.


Mounting on a Sterile Adapter Plate

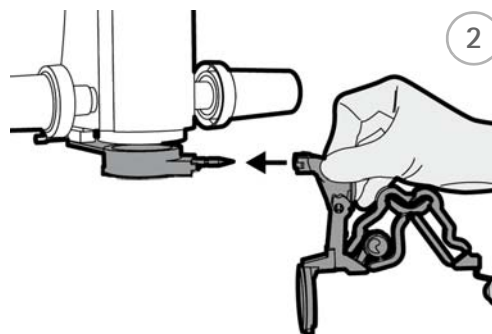- ➔ Put the sterile cap onto the knurled screw.
- ➔ Slide the sterile adapter plate into the dovetail mount.
Do not touch the microscope while doing this.

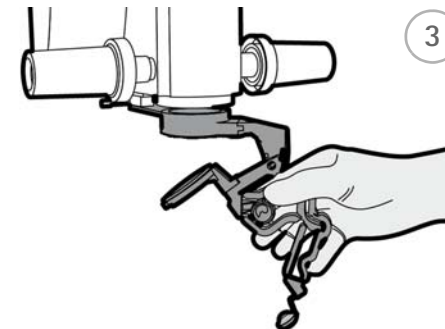- ➔ Using a minimum of force, plug the BIOM® ready onto the adapter plate until it reaches the stop. You must overcome a detent which locks the connection.

- ➔ To swing in the BIOM® ready, lift the bottom part a little.

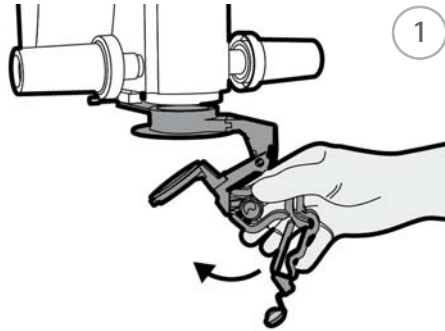
Caution

Risk to the patient if the BIOM® ready is inserted incorrectly.
If you do not insert the BIOM® ready correctly, you cannot use it safely and you will get a decentered fundus image.


- ➔ Make sure that the BIOM® ready has been inserted up to the limit stop.
-

Function Test Under Sterile Conditions

- Make sure that all components are present and are sterile.
- Clap the BIOM® ready against the adapter plate. It is equipped with a housing with swivel mechanism for 90° and a locking mechanism for this purpose.
- Make sure that the housing body can be tipped without resistance.
- Check the function of the focusing knob.

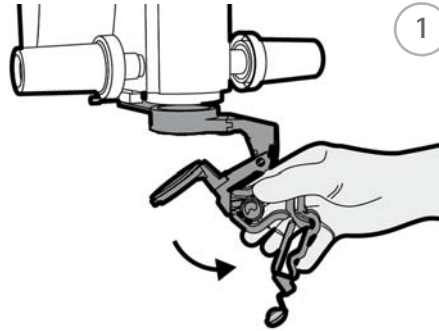Swing to Standby Position

- During the extra-ocular surgery phases, swing the BIOM® ready out of the beam path to its standby position.
- To swing out the BIOM® ready, lift the bottom part.

Make the Basic Settings at the Microscope

- Make the basic settings at the microscope in accordance with the manufacturer's specifications.
- Adjust the microscope to the anterior section of the eye and perform the surgical steps with microscope illumination, including putting in the infusion.

Instructions for Use

Caution

Risk of eye damage if the working distance between the BIOM® ready and the patient changes.

When the BIOM® ready is swung into the working position (into the beam path), the following instructions must be followed:

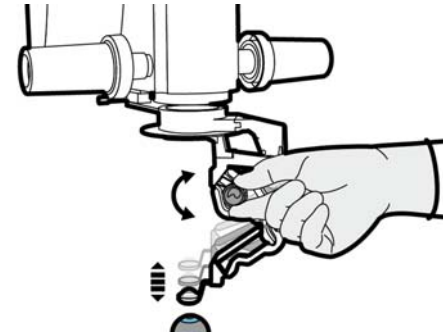
- **Do not** use the coarse adjustment knob at the microscope stand to adjust the height.
- **Do not** adjust the height of the stand arm, either by motor or manually, over the surgical area.
- **Do not** change the patient's position by adjusting the height of the OR- table.

Risk to the patient as a result of uncontrolled head movements of the patient.

- ➔ Take suitable measures to prevent uncontrolled head movements of the patient during the surgical procedure.

- ➔ Avoid any contact of the front lens with the cornea. If the lens touches the cornea, the image will become blurred immediately.
- ➔ In an extreme emergency, e.g. uncontrolled downward movement of the microscope:
 - Stop by pressing the Emergency Off button.
 - Pull / move the complete surgical microscope upwards.
 - Swing the BIOM® ready out of the beam path.
- ➔ If contact with the eye has occurred: Swing out the BIOM® ready, or lift the bottom part of the BIOM® ready to make it easier to clean the optics, so that you can clean the lens with a sterile swab. Clean the front lens with care, as it can easily break off.
- ➔ Make sure that the cornea is adequately wetted with a suitable solution. This prevents damage to the cornea and ensures optimal observation.

Focusing Instructions

You can manually focus the BIOM® ready with the focusing knob.

Heed the following points when focusing with the BIOM® ready:

- The BIOM® ready is only needed for visualization of the posterior eye section.
- ➔ After performing the surgical steps at the front section and outside of the eye, leave the microscope at this position (height).
- ➔ Before swinging it into the working position, make sure that the BIOM® ready has been set to the shortest length.
- ➔ Before beginning the focusing process, check the distance from the front lens to the patient's eye.
- ➔ Monitor the distance from the front lens to the patient's eye during the focusing process.
- ➔ Avoid the front lens from coming into contact with the patient's eye.
- ➔ While focusing with the BIOM® ready, make sure that
 - The working height of the microscope is **not** adjusted.
 - The focusing function of the microscope is **not** used.

Visualization of the Posterior Section of the Eye

After all preparations for surgery in the posterior section of the eye have been completed, proceed as follows without changing the microscope position:

- Use a suitable endo-illumination.
- Swing the BIOM® ready into the beam path of the microscope. To do so, lift the bottom part of the BIOM® ready and only release it when the BIOM® ready is at the final swung-in position.
- Switch off the microscope illumination.
- Activate the SDI® to reverse the total image reversal.

Dismounting and Disposal after Surgery

After the surgery, dismount the BIOM® ready and dispose of it in the proper manner, in accordance with the applicable hygiene regulations.

Caution

Risk to patient if the BIOM® ready is dismounted incorrectly

- Do not dismount the BIOM® ready over the patient.

- Do not dismount the BIOM® ready over the patient.
- Loosen the knurled screw; see page 16.
- Pull the BIOM® ready along with the adapter plate off the dovetail mount; see page 16.
- Pull the BIOM® ready off the adapter plate.
- Pull the sterile cap off the knob at the SDI®; see page 15.
- Remove the sterile cover from the adapter plate, if applicable.
- Pull the sterile cap off the knurled screw, if applicable; see page 16.
- Dispose of the disposable parts in accordance with the hospital's hygiene regulations and legal regulations.

Storage

→ Heed the following information.

Type of packaging	Storage
Cardboard packaging	<ul style="list-style-type: none"> ■ Unopened or sealed dust-tight ■ In a dark room ■ Ambient conditions: See page 3 ■ Storage duration: See the expiry date on the packaging
Non-sterile outer packaging	<ul style="list-style-type: none"> ■ Protected storage (e.g. in cabinets or drawers): 6 Months, but no later than the expiry date ■ Unprotected storage (e.g. on shelves): Use within 48 hours
Sterile inner packaging	<ul style="list-style-type: none"> ■ For immediate use

Troubleshooting

Caution

Risk to the patient if the BIOM® ready is faulty.

→ If a fault occurs that you cannot rectify by following the instructions below, dispose of the BIOM® ready in accordance with the hospital's hygiene regulations and legal regulations

Fault	Possible Cause	Remedy
Adapter or dovetail mount is wobbling	<ul style="list-style-type: none"> ■ Locking screw is too loose ■ Screws are loose 	<ul style="list-style-type: none"> ■ Tighten the locking screw hand-tight ■ Tighten the screws with a suitable screwdriver
Cropped or decentered image	<ul style="list-style-type: none"> ■ The SDI®, other components, or the BIOM® ready adapter have been mounted skewed 	<ul style="list-style-type: none"> ■ Re-mount correctly
Dull image	<ul style="list-style-type: none"> ■ Optical surfaces are dirty ■ Optical surfaces have been damaged by mechanical means ■ The front lens is touching the patient's eye ■ The patient's cornea is too dry 	<ul style="list-style-type: none"> ■ Clean the optical surfaces ■ Select the correct work setting, clean the front lens ■ Wet the cornea regularly with a suitable solution
Blurred image	<ul style="list-style-type: none"> ■ The BIOM® ready is maladjusted ■ The BIOM® ready model is not compatible with the microscope's lens 	<ul style="list-style-type: none"> ■ Focus the BIOM® ready in accordance with the instructions ■ Check the BIOM® ready model based on the reduction lens and if applicable, use the other BIOM® ready model
Insufficient fundus observation	<ul style="list-style-type: none"> ■ The front lens is too far away from the patient's eye ■ The microscope magnification has been set too high 	<ul style="list-style-type: none"> ■ Carefully reduce the distance with the microscope's focus ■ Reduce the microscope's magnification
Strong reflections on the patient's eye or lens	<ul style="list-style-type: none"> ■ The microscope illumination is switched on 	<ul style="list-style-type: none"> ■ Switch off the illumination; use intraocular illumination only

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OCULUS Surgical Inc.