





## OCULUS HD Disposable LenZ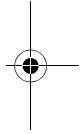

für den Einmalgebrauch, für ZEISS RESIGHT® 500 und 700  
disposable, for ZEISS RESIGHT® 500 and 700  
desechable para un solo uso, para ZEISS RESIGHT® 500 y 700  
jetable, pour ZEISS RESIGHT® 500 et 700  
monouso, per ZEISS RESIGHT® 500 e 700  
para uma utilização, ZEISS RESIGHT® 500 e 700  
Для одноразового пользования, для ZEISS RESIGHT® 500 и 700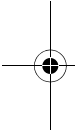

Montageanleitung de, Seite 7  
Assembly instructions en, page 11  
Instrucciones de montaje es, página 15  
Notice de montage fr, page 19  
Istruzioni per il montaggio it, pagina 23  
Instruções de montagem pt, página 27  
Руководство по монтажу ru, стр. 31

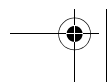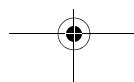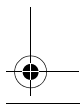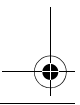


## OCULUS HD Disposable LenZ

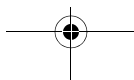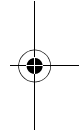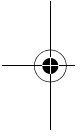
## Symbole / Symbols / Símbolos / Symboles / Simboli / Símbolos / Символы

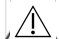
	<p>Hersteller Manufacturer Fabricante Fabricant Produttore Fabricante Производитель</p>		<p>Herstellungsdatum Manufacturing date Fecha de fabricación Date de fabrication Data di produzione Data de fabricaço Дата производства</p>
	<p>Nicht mit einem Messer öffnen Do not open with a knife No utilice cuchillo para abrir Ne pas ouvrir avec un couteau Non aprire con il coltello Não abra com uma faca! Не открывать ножом</p>		<p>Verwendbar bis Use by Fecha de caducidad À utiliser avant Utilizzabile fino al Utilizável até Использовать до</p>
	<p>Bei beschädigter Verpackung nicht verwenden Do not use when packaging is damaged No utilizar si el embalaje está dañado Ne pas utiliser si l'emballage est endommagé Non utilizzarlo se la confezione è danneggiata Com embalagem danificada não utilizar Не использовать в случае повреждения упаковки</p>		<p>Temperaturbegrenzung Lagerung Storage temperature limit Temperaturas límite de almacenamiento Températures de stockage admissibles Limite di temperatura per la tenuta in magazzino Limite de temperatura armazenamento Ограничение температуры хранения</p>
	<p>Relative Luftfeuchte / relative air humidity / humidité relative / umidità atmosferica relativa / humedad relativa humidade do ar relativo относительная влажность</p>		<p>Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление</p>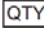

OCULUS HD Disposable LenZ

	<p>Trocken aufbewahren Keep in a dry place Conservar seco Conserver au sec Luogo asciutto Guardar em lugar seco <b>Сухое место / сухо сохраняют</b></p>		<p>Einmal verwenden Single use De un solo uso. Utiliser une seule fois. Monouso Usar uma vez <b>Использовать один раз</b></p>
	<p>steriles Produkt, einmal verwenden Sterile product, single use Producto estéril de un solo uso Produit stérilisé, à n'utiliser qu'une seule fois Prodotto sterile, usa e getta Produto esterilizado, usar uma vez <b>Стерильное изделие одноразового применения</b></p>		<p>CE Kennzeichnung mit benannter Stelle CE marking with notified body Cód. de pedido (cód. de artículo) Marcado CE con organismo notificado Marquage CE avec identification de l'organisme notifié Marchio CE con l'organismo notificato Marcação CE com organismo notificado <b>Маркировка CE с указанием уполномоченного органа</b></p>
	<p>Stückzahl Quantity Nº de unidades Quantité N. di pezzi... N.º de peças <b>Количество</b></p>		<p>Sterilisiert mit Ethylenoxid Sterilized with ethylene oxide Esterilizado con óxido de etileno Stérilisé avec de l'oxyde d'éthylène Sterilizzato con ossido di etilene Esterilizado com óxido de etileno <b>Стерилизация этиленоксидом</b></p>

OCULUS HD Disposable LenZ





	<p>Gebrauchsanweisung beachten          Observe instruction sheet          Siga las instrucciones de uso          Observer la notice d'utilisation          Osservare le Istruzioni per l'uso          Observar as instruções de utilização  <b>Следуйте инструкции по эксплуатации</b></p>		<p>Achtung          Attention          Atención          Attention          Attenzione          Atenção  <b>Внимание</b></p>
	<p>Enthält kein Naturkautschuklatex          Does not contain natural rubber latex          No contiene látex de caucho natural          Ne contient pas de latex de caoutchouc naturel          Non contiene lattice di gomma naturale          Não contém látex de borracha natural  <b>Не содержит латекс из натурального каучука</b></p>		<p>Bestellnummer (Artikelnummer)          Order number (article number)          Cód. de pedido (cód. de artículo)          N° de commande (n° d'article)          Numero di ordinazione (numero dell'articolo)          Número de encomenda (número do artigo)  <b>Номер заказа (номер артикула)</b></p>
	<p>Losnummer/Chargennummer          Lot number/batch number          Lote n°          N° du lot/n° de la charge          N. lotto/N. partita          Número do lote/número da carga  <b>Номер партии/Серийный номер</b></p>		<p>USA: verschreibungspflichtig          USA: Only available by prescription          EE. UU.: con receta médica          USA : uniquement sur ordonnance          USA: da vendersi dietro presentazione di ricetta medica          USA: receita obrigatória  <b>США: отпускается по рецепту</b></p>

## OCULUS HD Disposable LenZ


**Umgebungsbedingungen / Ambient conditions / Condiciones ambientales / Conditions ambiantes / Condizioni ambientali / Condições do ambiente / Условия окружающей среды**

	Temperatur / temperature / temperatura / température / temperatura / temperatura / температура	Relative Luftfeuchte / relative air humidity / humidité relative / umidità atmosferica relativa / humedad relativa / humidade do ar relativo / относительная влажность	Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
Transport / transport / transport / trasporto / transporte / transporte / перевоз	-10°C – +50°C (-14°F – +122°F)	10 % – 95 %	500 hPa – 1060 hPa
Lagerung / storage / stockage / stoccaggio / almacenamiento / armazenagem / хранение	+5°C – +50°C (41°F – +122°F)	10 % – 95 %	700 hPa – 1060 hPa
Gebrauch / use / usage / uso / uso / uso / употребление	+10°C – +40°C (+50°F – +104°F)	30 % – 90 %	800 hPa – 1060 hPa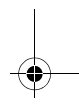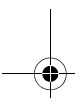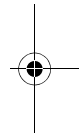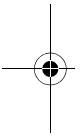

OCULUS HD Disposable LenZ

**Übersicht / Overview / Vista general / Aperçu /  
Visione d'insieme / Vista geral / Обзор**

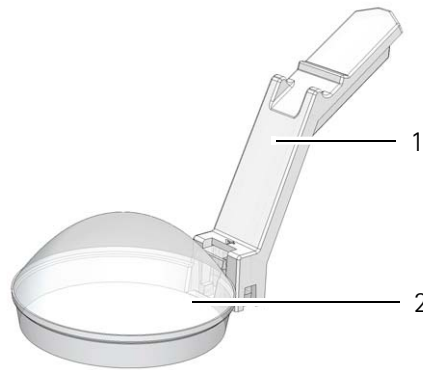OCULUS HD Disposable LenZ

1:

*Lupenhalter*

*Lens holder*

*Porta lupa*

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*Porta lente*

*Suporte de lupa*

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## Assembly instructions

The OCULUS HD Disposable LenZ is an accessory for the use with the ZEISS RESIGHT®. These instructions only describe the assembly of the OCULUS HD Disposable LenZ. Further information about the ZEISS RESIGHT® is included in the instruction of use of the ZEISS RESIGHT®.

### Scope of delivery

Components	Order number
OCULUS HD Disposable LenZ for ZEISS RESIGHT®	54416
Assembly instructions OCULUS HD Disposable LenZ	G/54416/.../en

The manufacturer reserves the right to make changes in the scope of delivery due to further technical development.

- ➔ If you discover transport damage in your delivery, please notify the transporting company immediately.

### Safety notes

- ➔ Please read the assembly instructions carefully.
- ➔ Only work with the OCULUS HD Disposable LenZ when you have understood the assembly instructions.
- ➔ Take care to keep the assembly instructions near the OCULUS HD Disposable LenZ.
- ➔ Carefully follow the instruction of use of the ZEISS RESIGHT®, in particular the safety notices.
- ➔ Never use a damaged OCULUS HD Disposable LenZ.
- ➔ Do not reuse the OCULUS HD Disposable LenZ.
- ➔ Ensure that the sterile packaging and the OCULUS HD Disposable LenZ are not damaged.
- ➔ Never attach or remove the OCULUS HD Disposable LenZ above a patient.
- ➔ Observe the legal provisions valid in your country as well as the hygiene and disposal regulations of your hospital or clinic.

### Proper use

The OCULUS HD Disposable LenZ allows visualization of the ocular fundus as well as of vitreous body and retina structures during vitreo-retinal surgeries. It is intended for single use on the ZEISS RESIGHT®. Use the OCULUS HD Disposable

Assembly Instructions OCULUS HD Disposable LenZ

en

LenZ under strictly sterile conditions. It is not possible to resterilize the OCULUS HD Disposable LenZ.

➔ Observe the safety notes listed above.

**Storage**

Type of packaging	Storage
Cardboard box	<ul style="list-style-type: none"> <li>■ Unopened or closed dust-tight</li> <li>■ In a dark room</li> <li>■ Ambient conditions: refer to page 4</li> <li>■ Storage duration: see "Use by" date on the package</li> </ul>
Non-sterile outer packaging	<ul style="list-style-type: none"> <li>■ Protected storage (e.g. in cabinets or drawers): 6 months, but not longer than the expiration date</li> <li>■ Unprotected storage (e.g. on shelves): Use within 48 hours</li> </ul>
Sterile inner packaging	<ul style="list-style-type: none"> <li>■ For immediate use</li> </ul>

➔ Before taking a package out of an open box, clean all dust off the cardboard box.

➔ Do **not** refill the box.

**Unpacking**

**Caution**

Patient hazard: Contamination caused by faulty unpacking  
 The OCULUS HD Disposable LenZ is wrapped in sterile packaging. If the OCULUS HD Disposable LenZ is unpacked incorrectly, it may become contaminated.

➔ When unpacking, please take care that the OCULUS HD Disposable LenZ remains sterile.

The OCULUS HD Disposable LenZ is wrapped in dual packaging.

- Non-sterile outside packaging
- Sterile inside packaging

Non-sterile personnel:

- Check packaging for damage. If the packaging is damaged, the OCULUS HD Disposable LenZ must not be used.
- Remove the outside packaging. Do not touch the inside packaging.

Sterile personnel


- Take the OCULUS HD Disposable LenZ out of the sterile packaging.

### Installation with sterile assistance

**Caution** Patient hazard caused by incorrect installation

- Never attach the OCULUS HD Disposable LenZ above a patient.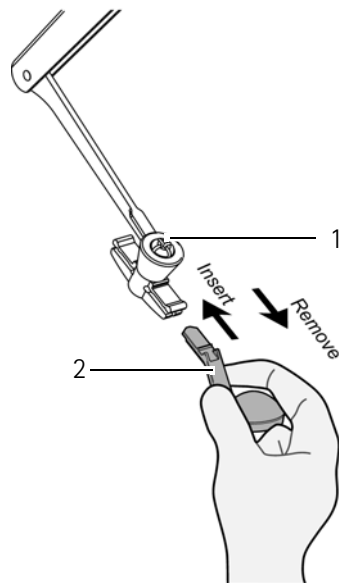

- Take the OCULUS HD Disposable LenZ from the packaging without touching the surfaces of the magnifier.
- Hold the OCULUS HD Disposable LenZ (2) on its lens holder.
- Insert the HD Disposable LenZ into the holder of the ZEISS RESIGHT® (1). It must engage (direction of the arrow: Insert).

### Dismantling and disposal


After use, the OCULUS HD Disposable LenZ must be removed and disposed of.

#### Caution

Risk of personal injury caused by falling, contaminated OCULUS HD Disposable LenZ Set

- Never remove the OCULUS HD Disposable LenZ above a patient.
- After surgery, dispose of the disposable parts according to the hygiene regulations of your hospital and the legal requirements.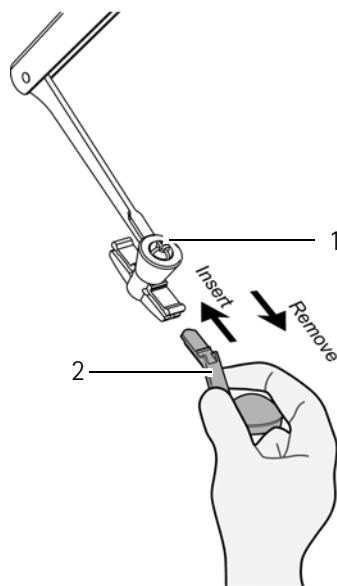

- Remove the HD Disposable LenZ out of the holder of the ZEISS RESIGHT® (1) (direction of the arrow: Remove).
- Dispose of the OCULUS HD Disposable LenZ (2) according to the hygiene regulations of your hospital and the legal requirements.
- Ensure that the ZEISS RESIGHT® is conditioned according to the Zeiss manual.

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OCULUS Optikgeräte GmbH
OCULUS Surgical Inc.