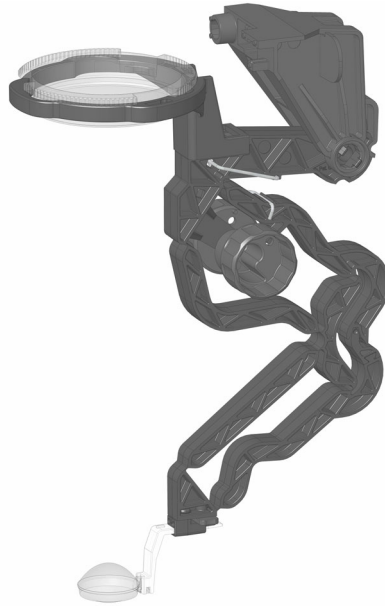
BIOM® ready+

für den Einmalgebrauch, für f = 175 mm oder f = 200 mm
 disposable, for f = 175 mm or f = 200 mm
 desechable para un solo uso, para f = 175 mm ó f = 200 mm
 jetable, pour f = 175 mm ou f = 200 mm
 monouso, per f = 175 mm o f = 200 mm
 para uma utilização, para f = 175 mm ou f = 200 mm
 Для одноразового пользования, для f = 175 мм или f = 200 мм

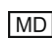
Gebrauchsanweisung de, Seite 5
Instruction Manual en, page 13
Manual de Instrucciones es, página 21
Mode d'emploi fr, page 29
Instruzioni per l'uso it, pagina 37
Manual de instruções pt, página 45
Руководство по эксплуатации ru, стр. 53

G/54419/IN Rev02
 Freigabe: 21.11.2022

Symbole / Symbols / Símbolos / Symboles / Simboli / Símbolos / Символы

	Hersteller Manufacturer Fabricante Fabricant Produttore Fabricante Производитель		Herstellungsdatum Manufacturing date Fecha de fabricación Date de fabrication Data di produzione Data de fabricaço Дата производства
	Nicht mit einem Messer öffnen Do not open with a knife No utilice cuchillo para abrir Ne pas ouvrir avec un couteau Non aprire con il coltello Não abra com uma faca! Не открывать ножом		Verwendbar bis Use by Fecha de caducidad À utiliser avant Utilizzabile fino al Utilizável até Использовать до
	Bei beschädigter Verpackung nicht verwenden Do not use when packaging is damaged No utilizar si el embalaje está dañado Ne pas utiliser si l'emballage est endommagé Non utilizzarlo se la confezione è danneggiata Com embalagem danificada não utilizar Не использовать в случае повреждения упаковки		Temperaturbegrenzung Lagerung Storage temperature limit Temperaturas límite de almacenamiento Températures de stockage admissibles Limite di temperatura per la tenuta in magazzino Limite de temperatura armazenamento Ограничение температуры хранения

	Relative Luftfeuchte / relative air humidity / humidité relative/ umidità atmosferica relativa / humedad relativa humidade do ar relativo относительная влажность		Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
	Trocken aufbewahren Keep in a dry place Conservar seco Conserver au sec Luogo asciutto Guardar em lugar seco Сухое место / сухо сохраняют		einmal verwenden Use once De un solo uso. Utiliser une seule fois. Monouso Usar uma vez Использовать один раз
	steriles Produkt, einmal verwenden Sterile product, use once Producto estéril de un solo uso produit stérilisé, à n'utiliser qu'une seule fois Prodotto sterile, usa e getta Produto esterilizado, usar uma vez Стерильное изделие одноразового применения		CE Kennzeichnung mit benannter Stelle CE marking with notified body Cód. de pedido (cód. de artículo) Marcado CE con organismo notificado Marquage CE avec identification de l'organisme notifié Marchio CE con l'organismo notificato Marcação CE com organismo notificado Маркировка CE с указанием уполномоченного органа
QTY	Stückzahl Quantity Nº de unidades Quantité N. di pezzi... N.º de peças Количество		Sterilisiert mit Ethylenoxid Sterilized with ethylene oxide Esterilizado con óxido de etileno Stérilisé avec de l'oxyde d'éthylène Sterilizzato con ossido di etilene Esterilizado com óxido de etileno Стерилизация этиленоксидом

	Gebrauchsanweisung beachten Observe instruction sheet Siga las instrucciones de uso Observer la notice d'utilisation Osservare le Istruzioni per l'uso Observar as instruções de utilização Следуйте инструкции по эксплуатации		Achtung Attention Atención Attention Attenzione Atenção ВНИМАНИЕ
	Enthält kein Naturkautschuklatex Does not contain natural rubber latex No contiene látex de caucho natural Ne contient pas de latex de caoutchouc naturel Non contiene lattice di gomma naturale Não contém látex de borracha natural Не содержит латекс из натурального каучука		Bestellnummer (Artikelnummer) Order number (article number) Cód. de pedido (cód. de artículo) Nº de commande (nº d'article) Numero di ordinazione (numero de l'articolo) Número de encomenda (número do artigo) Номер заказа (номер артикула)
	Losnummer/Chargennummer Lot number/batch number Lote nº Nº du lot/nº de la charge N. lotto/N. partita Número do lote/número da carga Номер партии/Серийный номер	R_x ONLY	USA: verschreibungspflichtig USA: Only available by prescription EE. UU.: con receta médica USA : uniquement sur ordonnance USA: da vendersi dietro presentazione di ricetta medica USA: receita obrigatória США: отпускается по рецепту
	Medizinprodukt Medical device Dispositivo médico Dispositif médical Dispositivo medico Dispositivo médico медицинское изделие		

Technische Daten / Technical data / Datos técnicos /
 Caractéristiques techniques / Dati tecnici / Dados técnicos / Технические
 характеристики

Breite / Width / Anchura / Largeur / Larghezza / Largura / Ширина	58 mm (2,28 in)
Tiefe / Depth / Profundidad / Profondeur / Profondità / Profundidade / Глубина	116 mm (4,57 in)
Höhe, maximal / Height, maximum / Altura máxima / Hauteur, maximale / Itezza massima / Altura, máxima / Высота, макс. ■ f = 200 mm ■ f = 175 mm	180 mm (7,09 in) 160 mm (6,30 in)
Hub Sicherheitseinschub, minimal / Travel of safety rod, minimum / Carrera mínima chasis de seguridad / Déplacement de sécurité, minimal / Corsa minima innesto di sicurezza / Inserção de segurança curso, mínimo / Ход предохранительного выдвижного шасси, мин.	~ 27 mm (1,06 in)
Verstellbereich / Adjustment range / Rango de ajuste / Domaine de réglage / Campo di regolazione / Área de ajuste / Диапазон регулировки	23 mm (0,91 in)
Gewicht / Weight / Peso / Poids / Peso / Peso / Вес ■ f = 200 mm ■ f = 175 mm	~ 46,5 g (1,64 oz) ~ 41,0 g (1,45 oz)

Umgebungsbedingungen/ Ambient conditions / Condiciones ambientales /
 Conditions ambiantes / Condizioni ambientali / Condições do ambiente /
 Условия окружающей среды

	Temperatur / temperature / temperatura / température / temperatura / temperatura / температура	Relative Luftfeuchte / relative air humidity / humidité relative / umidità atmosferica relativa / humedad relativa / humidade do ar relativo / относительная влажность	Luftdruck / air pressure / pression d'air / pressione atmosferica / presión atmosférica / pressão do ar / атмосферное давление
Transport / transport / transport / trasporto / transporte / transporte / перевоз	-10 °C – +50 °C (+14 °F – +122 °F)	10 % – 85 %	700 hPa – 1060 hPa
Lagerung / storag e/ stockage / stoccaggio / almacenamiento / armazenagem / хранение	+5 °C – +50 °C (+41 °F – +122 °F)	10 % – 95 %	700 hPa – 1060 hPa
Gebrauch / use / usage / uso / uso / uso / употребление	+10 °C – +40 °C (+50 °F – +104 °F)	30 % – 90 %	800 hPa – 1060 hPa

Mikroskope/ Microscopes/ Microscopios/ Microscope/ Microscopi/
Microscópios/ Микроскоп

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Instruction Manual

Scope of Delivery

Components	Order Number
BIOM® ready+ for f = 175 mm	54419
BIOM® ready+ for f = 200 mm	54420
Including	
<ul style="list-style-type: none"> ■ 2 sterile covers for adapter plates with different diameters for BIOM® ready+ ■ Sterile cap for the knurled screw at the dovetail mount ■ Sterile cap for the knob of the SDI® 	
Instruction Manual BIOM® ready+	G/54419/INxxxx

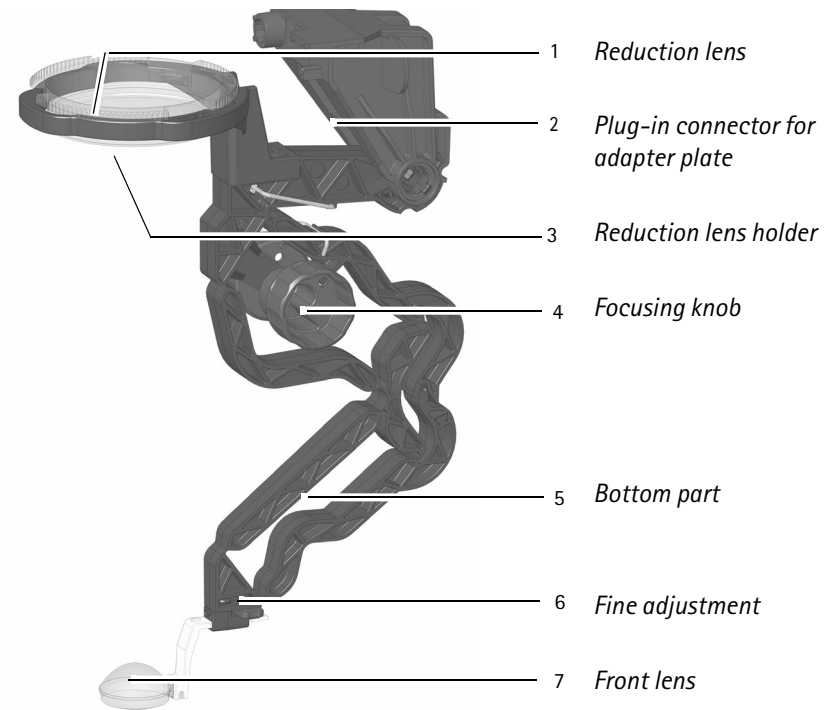
Scope of delivery subject to change due to ongoing technical development.

- ➔ If you discover transport damage upon delivery, please report this to the shipping company immediately.


Safety Instructions

- ➔ Read through the Instruction Manual carefully.
- ➔ Heed the following safety instructions.
- ➔ Keep the Instruction manual in a safe place in the vicinity of the unit.
- ➔ Make sure that the BIOM® ready+ is only used by trained physicians and OR employees, who, due to their knowledge, training and practical experience, can guarantee proper use.
- ➔ Heed the instruction manuals and safety instructions for the surgical microscope and any other equipment you are using.
- ➔ Before use: Get OCULUS or an authorized dealer to train you in the operation of the BIOM® ready+.
- ➔ Make sure that the packaging and the BIOM® ready+ are undamaged and that the BIOM® ready+ is sterile.
- ➔ Before use, pay attention to the storage conditions; see page 19.
- ➔ Do not mount and dismount the BIOM® ready+ over the patient.
- ➔ Comply with the legal regulations in force in your country and with the hygiene and disposal regulations of the hospital or the clinic.

Device Components

Accessories

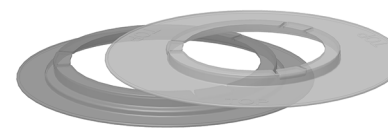
Sterile cover, colourless, for adapter plates with standard inner diameter of approx. 55 mm. Article no.: 55423, 55424, 55425, 55432, 55431.

Sterile cover, blue, for adapter plates with larger inner diameter of approx. 70 mm. Article no.: 55426, 55429

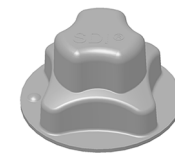Sterile cap for the knurled screw

Sterile cap for the knob

Proper Use

This Binocular Indirect Ophthalmo-Microscope (BIOM® ready+) is used for non-contact observation of surgical procedures in the posterior section of the eye.

The BIOM® ready+ is intended for single use only. It may only be used in sterile conditions. It cannot be re-processed for reuse.

The BIOM® ready+ is intended for use in conjunction with compatible surgical microscopes in hospitals, clinics and other medical institutions.

The BIOM® ready+ is available in models for a focal length of $f = 175$ mm or for a focal length of $f = 200$ mm. It has a wide-angle front lens and a reduction lens.

The maximum observation of the fundus is approx. 130° , the working distance (front lens - cornea) is approx. 3-4 mm.

The surgical microscopes must be designated as adaptable by OCULUS Optikgeräte GmbH, see page 4.

Mode of Operation

In conjunction with an SDI® (Stereoscopic Diagonal Inverter) for erecting the image, the BIOM® ready+ enables non-contact, wide-angle observation of the fundus and vitreous cavity.

The combination of a surgical microscope and the optical components of the BIOM® ready+ allows the vitreous cavity to be observed under stereoscopic conditions. As an indirect ophthalmo-microscope, the BIOM® ready+ works without touching the cornea during the surgical procedure.

The patient's eyeball can be moved freely during the procedure. Peripheral fundus sections are thus easy to see.

The optical system of the BIOM® ready+ consists of a reduction lens and a front lens. The reduction lens ensures a more or less constant distance between the patient's eye and the surgical microscope when swinging the BIOM® ready+ in and out.

The position of the reduction lens with respect to the surgical microscope is fixed.

The BIOM® ready+ image is focused by adjusting the height of the front lens. The bottom part is used to adjust the height of the front lens. The distance between the surgical microscope and the front lens is set manually using the focusing knob at the BIOM® ready+.

Unpacking

Caution

Risk to the patient: Contamination if unpacked incorrectly. The BIOM® ready+ is sterile-packed. If you unpack the BIOM® ready+ incorrectly, it can become unsterile and thus unusable.

- ➔ When unpacking the BIOM® ready+, make sure that it stays sterile.
- ➔ Before use, pay attention to the storage conditions; see page 19.

The BIOM® ready+ is double-packed

- Non-sterile outer packaging
- Sterile inner packaging

Please heed the following instructions:

Non-sterile personnel:

- ➔ Check the outer packaging for damage. If the packaging is damaged, you must not use the BIOM® ready+.
- ➔ Check whether the focal length of your microscope lens matches the BIOM® ready+ model.
- ➔ Open the outer package. Take care **not** to touch the inner packaging as you do so.
- ➔ Make sure that the sterile BIOM® ready+ does not become contaminated and thus unsterile.

Sterile personnel:

- ➔ Remove the BIOM® ready+ from its sterile packaging.

Mounting Information

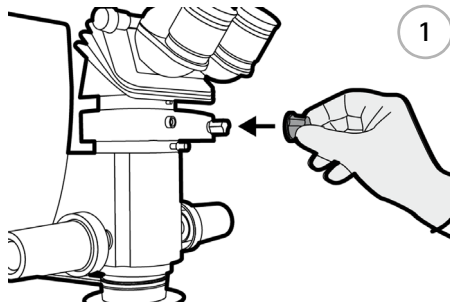
Caution


Risk to the patient if the BIOM® ready+ is mounted incorrectly

- ➔ Do not mount the BIOM® ready+ over the patient.
- ➔ Make sure that the BIOM® ready+ is installed by a person who has followed appropriate disinfection procedures.


Mounting on a Non-Sterile Adapter Plate

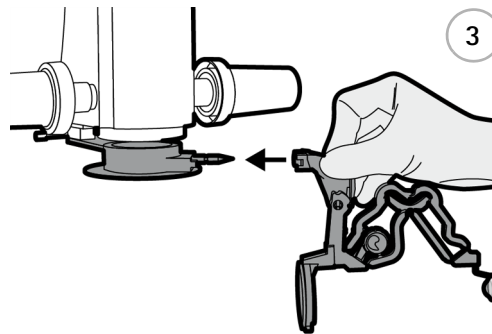
→ Put the sterile cap into place at the SDI®.

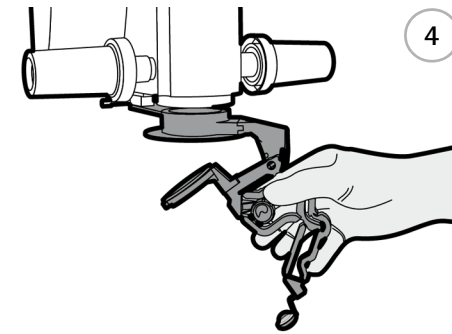→ Using a minimum of force, plug the BIOM® ready+ onto the adapter plate until it reaches the stop. You must overcome a detent which locks the connection.

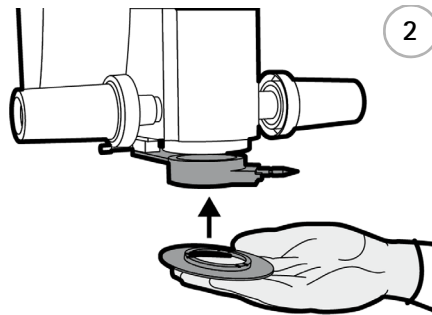→ To swing in the BIOM® ready+, lift the bottom part a little.

→ Place the matching sterile cover on the adapter plate. Do not touch any microscope parts or the adapter plate. The sterile disc must be pressed on with the labelling "TOP" facing upwards, in the direction of the adapter plate.

Caution

Danger to patients due to contamination

→ Do not touch the non-sterile microscope when assembling the BIOM® ready+ and the sterile cover.

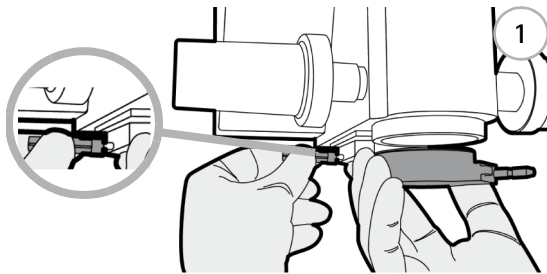
Risk to the patient if the BIOM® ready+ is inserted incorrectly.

If you do not insert the BIOM® ready+ correctly, you cannot use it safely and you will get a decentered fundus image.


→ Make sure that the BIOM® ready+ has been inserted up to the limit stop.

→ Check the BIOM® ready+ for tight fit.


Mounting on a Sterile Adapter Plate

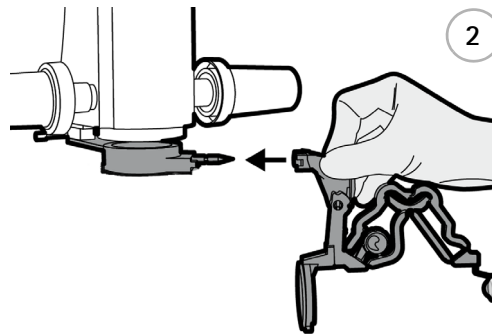- Put the sterile cap onto the knurled screw.
- Push the sterile adapter plate into the dovetail ring up to the stop.
- Firmly tighten the knurled screw using your hand.
- Make sure that the adapter plate has been pushed in up to the stop and is free of play.

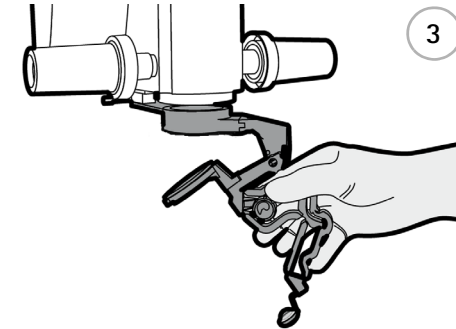- Using a minimum of force, plug the BIOM® ready+ onto the adapter plate until it reaches the stop. You must overcome a detent which locks the connection.

- To swing in the BIOM® ready+, lift the bottom part a little.

Caution

Danger to patients due to contamination

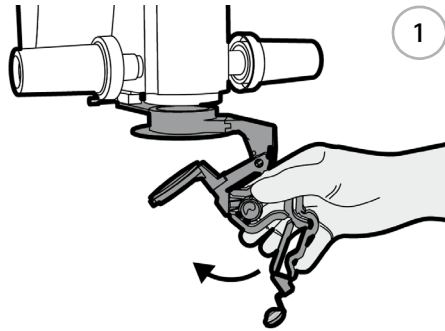
- Do not touch the unsterile microscope during installation.

Risk to the patient if the BIOM® ready+ is inserted incorrectly.

If you do not insert the BIOM® ready+ correctly, you cannot use it safely and you will get a decentered fundus image.


- Make sure that the BIOM® ready+ has been inserted up to the limit stop.

Function Test Under Sterile Conditions

- Make sure that all components are present and are sterile.
- Clap the BIOM® ready+ against the adapter plate. It is equipped with a housing with swivel mechanism for 90° and a locking mechanism for this purpose.
- Make sure that the housing body can be tipped without resistance.
- Check the function of the focusing knob.

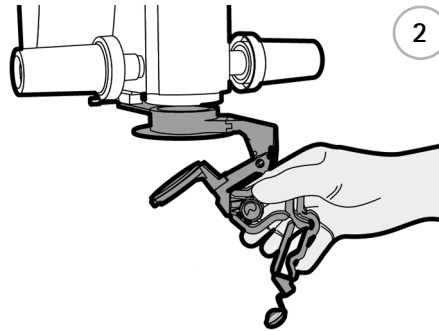Swing to Standby Position

- During the extra-ocular surgery phases, swing the BIOM® ready+ out of the beam path to its standby position.
- To swing out the BIOM® ready+, lift the bottom part.

Make the Basic Settings at the Microscope

- Make the basic settings at the microscope in accordance with the manufacturer's specifications.
- Adjust the microscope to the anterior section of the eye and perform the surgical steps with microscope illumination, including putting in the infusion.

Instructions for Use

Caution

Risk of eye damage if the working distance between the BIOM® ready+ and the patient changes.

When the BIOM® ready+ is swung into the working position (into the beam path), the following instructions must be followed:

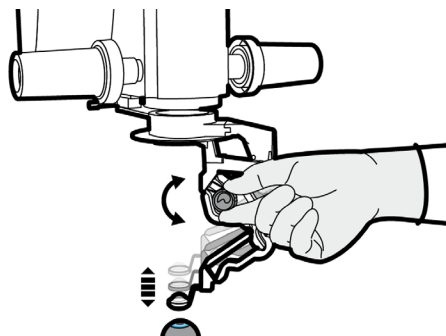
- Do not use the coarse adjustment knob at the microscope stand to adjust the height.
- Do not adjust the height of the stand arm, either by motor or manually, over the surgical area.
- Do not change the patient's position by adjusting the height of the OR- table.

Risk to the patient as a result of uncontrolled head movements of the patient.

- Take suitable measures to prevent uncontrolled head movements of the patient during the surgical procedure.

- Avoid any contact of the front lens with the cornea.
If the lens touches the cornea, the image will become blurred immediately.
- In an extreme emergency, e.g. uncontrolled downward movement of the microscope:
 - Stop by pressing the Emergency Off button.
 - Pull / move the complete surgical microscope upwards.
 - Swing the BIOM® ready+ out of the beam path.
- If contact with the eye has occurred: Swing out the BIOM® ready+, or lift the bottom part of the BIOM® ready+ to make it easier to clean the optics, so that you can clean the lens with a sterile swab.
Clean the front lens with care, as it can easily break off.
- Make sure that the cornea is adequately wetted with a suitable solution. This prevents damage to the cornea and ensures optimal observation.

Focusing Instructions

You can manually focus the BIOM® ready+ with the focusing knob.

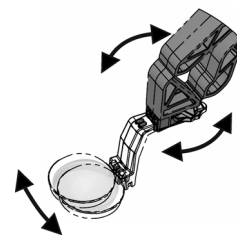
Heed the following points when focusing with the BIOM® ready+:

- The BIOM® ready+ is only needed for visualization of the posterior eye section.
- After performing the surgical steps at the front section and outside of the eye, leave the microscope at this position (height).
- Before swinging it into the working position, make sure that the BIOM® ready+ has been set to the shortest length.
- Before beginning the focusing process, check the distance from the front lens to the patient's eye.

- Monitor the distance from the front lens to the patient's eye during the focusing process.
- Avoid the front lens from coming into contact with the patient's eye.
- While focusing with the BIOM® ready+, make sure that
 - The working height of the microscope is **not** adjusted.
 - The focusing function of the microscope is **not** used.

Notes on centring

By means of fine adjustment, you can centre the front lens precisely.

- Use the fine adjustment mechanism to centre the front lens optimally

Visualization of the Posterior Section of the Eye

After all preparations for surgery in the posterior section of the eye have been completed, proceed as follows without changing the microscope position:

- Use a suitable endo-illumination.
- Swing the BIOM® ready+ into the beam path of the microscope. To do this, lift the lower part of the BIOM® ready+ and only release it slowly and carefully in the swivelled-in end position.
- Switch off the microscope illumination.
- Activate the SDI® to reverse the total image reversal.

Dismounting and Disposal after Surgery

After the surgery, dismount the BIOM® ready+ and dispose of it in the proper manner, in accordance with the applicable hygiene regulations.

Caution

Risk to patient if the BIOM® ready+ is dismounted incorrectly

- Do not dismount the BIOM® ready+ over the patient.

- Do not dismount the BIOM® ready+ over the patient.
- Loosen the knurled screw; see page 16.
- Pull the BIOM® ready+ along with the adapter plate off the dovetail mount; see page 16.
- Pull the BIOM® ready+ off the adapter plate.
- Pull the sterile cap off the knob at the SDI®; see page 15.
- In case the sterile cover of the adapter plate has been installed, remove the sterile cover.
- Pull the sterile cap off the knurled screw, if applicable; see page 16.
- Dispose of the disposable parts in accordance with the hospital's hygiene regulations and legal regulations.

Storage

→ Heed the following information.

Type of packaging	Storage
Cardboard packaging	<ul style="list-style-type: none"> ■ Unopened or sealed dust-tight ■ In a dark room ■ Ambient conditions: See page 3 ■ Storage duration: See the expiry date on the packaging
Non-sterile outer packaging	<ul style="list-style-type: none"> ■ Protected storage (e.g. in cabinets or drawers): 6 Months, but no later than the expiry date ■ Unprotected storage (e.g. on shelves): Use within 48 hours
Sterile inner packaging	<ul style="list-style-type: none"> ■ For immediate use

Troubleshooting

Caution

Risk to the patient if the BIOM® ready+ is faulty.

→ If a fault occurs that you cannot rectify by following the instructions below, dispose of the BIOM® ready+ in accordance with the hospital's hygiene regulations and legal regulations

Fault	Possible Cause	Remedy
Adapter or dovetail mount is wobbling	<ul style="list-style-type: none"> ■ Locking screw is too loose ■ Screws are loose 	<ul style="list-style-type: none"> ■ Tighten the locking screw hand-tight ■ Tighten the screws with a suitable screwdriver
Cropped or decentered image	<ul style="list-style-type: none"> ■ The SDI®, other components, or the BIOM® ready+ adapter have been mounted skewed 	<ul style="list-style-type: none"> ■ Re-mount correctly
Dull image	<ul style="list-style-type: none"> ■ Optical surfaces are dirty ■ Optical surfaces have been damaged by mechanical means ■ The front lens is touching the patient's eye ■ The patient's cornea is too dry 	<ul style="list-style-type: none"> ■ Clean the optical surfaces ■ Select the correct work setting, clean the front lens ■ Wet the cornea regularly with a suitable solution
Blurred image	<ul style="list-style-type: none"> ■ The BIOM® ready+ is maladjusted ■ The BIOM® ready+ model is not compatible with the microscope's lens 	<ul style="list-style-type: none"> ■ Focus the BIOM® ready+ in accordance with the instructions ■ Check the BIOM® ready+ version based on the labelling on the BIOM® ready+ and use a different BIOM® ready+ version if necessary.
Insufficient fundus observation	<ul style="list-style-type: none"> ■ The front lens is too far away from the patient's eye ■ The microscope magnification has been set too high 	<ul style="list-style-type: none"> ■ Carefully reduce the distance with the microscope's focus ■ Reduce the microscope's magnification
Strong reflections on the patient's eye or lens	<ul style="list-style-type: none"> ■ The microscope illumination is switched on 	<ul style="list-style-type: none"> ■ Switch off the illumination; use intraocular illumination only

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OCULUS Surgical Inc.