Super View® Hassan-Tornambe Disposable Contact Lens

REF 54012.000

Instructions for use

Contents: (1) Disposable, flat contact lens for vitrectomy

Indications for Use:

Allows visualization of the ocular fundus, vitreous and retinal structures during vitreoretinal surgery.

Performance Characteristics:

This disposable contact lens provides a non-inverted view of the vitreous cavity and retina. With a dilated pupil, a field of view up to 36 degrees is achievable. When moistened, the flange easily conforms and adheres to the cornea and sclera in order to stabilize the lens.

Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.

Warning:

- To be used only by a licensed physician in a method consistent with other direct image ophthalmic vitrectomy lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.

Directions:

Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found on the packaging.
2. After a visual check, open the outer pouch without touching the inner packaging tray.
3. Let the tray slide onto a sterile surface. Make sure not to contaminate the tray during this process.


Sterile operating nurse/surgtech

4. Flush cornea with ophthalmic solution.
5. While taking care not to touch the lens surface, hold the lens only by its edge and place a "tear drop" amount of ophthalmic lubricant on the curved underside of the contact lens.
6. Place on the cornea, curved side down and hold in place with a cotton swab for several seconds to displace any air bubbles and stabilize the lens on the eye.

Sterilization:

This item is delivered in sterile condition and is for **SINGLE USE ONLY**. After surgery, it should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

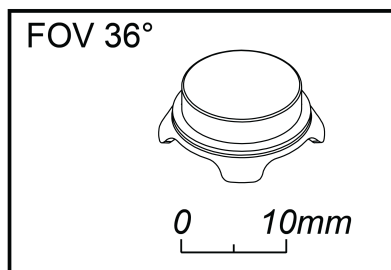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