Super View® Disposable Lens Set, PIVOT™

REF 54015 for BIOM® m/c
REF 54028 for BIOM® ml/cl

Instructions for Use

Contents:

- (1) PIVOT Assembly with Wide Field and Ultra HD Lenses
- (1) Correction Lens
- (2) Drive Belts (for use with BIOM® 3c, 4c, 5c)
- (1) Thumb Screw Cover (for dovetail)

Indications for Use:

Allows visualization of the ocular fundus, vitreous and retinal structures during vitreoretinal surgery.

Performance Characteristics:

This lens set consists of a disposable, rotatable lens holder providing two options for viewing of the vitreous cavity and retina. It features a wide field lens for viewing of the entire fundus and far periphery up to the ora serrata, and a Ultra HD wide field lens for a higher magnification view up to the mid-periphery. Also included in each lens set are a disposable correction lens, two drive belts (for use with motorized BIOM®) and a thumb screw cover for the OCULUS BIOM® wide angle viewing system.

Intended use:

Intended for use with the BIOM® Viewing System.

Objective Lens	BIOM®	BIOM® Optic Set (disposable)	REF
f=175mm	BIOM® 3 m/c, 4 m/c, 5m/c	Super View® Lens Set, Pivot f=175mm	54015
f=200mm	BIOM® 3 ml/cl, 4 ml/cl, 5ml/cl	Super View® Lens Set, Pivot f=200mm	54028

Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.

Warning:

- To be used only by a licensed physician in a method consistent with other wide-angle surgical ophthalmic lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.


Directions: Wide-angle lenses

Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found on the packaging.
2. After a visual check, open the outer pouch without touching the inner packaging tray.
3. Let the tray slide onto a sterile surface. Make sure not to contaminate the tray during this process.

Sterile operating nurse/surgtech

4. Open the sterile tray and remove contents as needed.
5. Hold the correction lens (Figure 1-A) by its flange and place it in the lens holder (Figure 1-B), so that the mounting ring tabs on the lens fit into the notches in the lens holder.
6. Turn the correction lens clockwise, passing through the resistance of the spring clip. The correction lens is now secure and will not fall out.

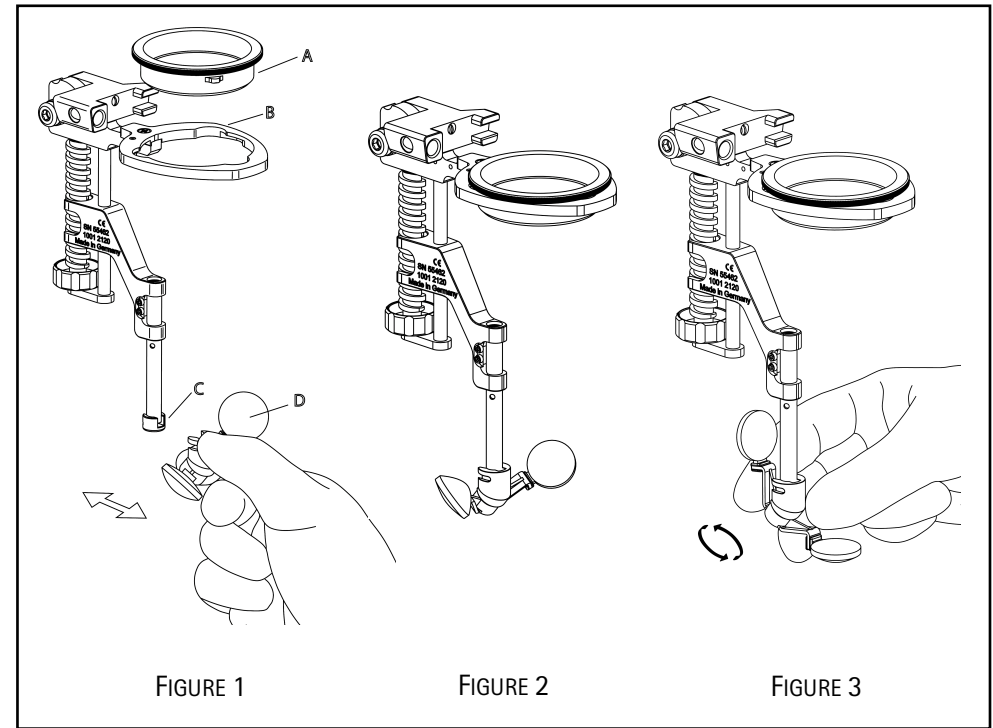7. Using your thumb and index finger, hold the PIVOT Assembly as shown in Figure 1-D without touching the lens surfaces. **FAILURE TO HOLD THE CLIP IN THIS MANNER MAY RESULT IN DAMAGE TO THE ASSEMBLY.** Mount the PIVOT by inserting the black plastic tab into the slot (Figure 1-C).
8. Rotate the desired lens into operating position (Figure 3). Exercise caution not to contact the patient when rotating lenses during the procedure.
9. To remove the PIVOT after use, rotate the lenses to the position shown in Figure 2. Grasp the body of the PIVOT and pull out and away from the BIOM®.
10. To remove the correction lens (Figure 1-A) after use, turn the lens counter-clockwise and carefully lift out of the lens ring holder.

Directions: Thumb screw cover

1. Grasp thumb screw cover by flat end and push over the dovetail thumb screw.
2. When tightening or loosening thumb screw, grasp cover by the round portion.
3. Remove cover after use by grasping flat end and pulling off of thumb screw.

Directions: Drive belts

Please refer to BIOM® manufacturers instructions for use.

Sterilization:

These items are delivered in a sterile condition and are for **SINGLE USE ONLY**. After surgery, they should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

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