Super View® Hassan Contact Lens Set

REF 54023.000

Instructions for use

- Contents:**
- (1) 0° flat contact lens for vitrectomy
 - (1) 30° flat contact lens for vitrectomy
 - (1) 50° flat contact lens for vitrectomy

Indications for Use:

Allows visualization of the ocular fundus, vitreous and retinal structures during vitreoretinal surgery.

Performance Characteristics:

This contact lens set provides disposable lens options for non-inverted viewing of the vitreous cavity and retina at different viewing angles. The 0 degree flat lens, 30 degree prismatic lens, and 50 degree prismatic lens are interchangeable within the lens ring, and the prismatic lenses can be rotated within the lens ring to view the desired sections of the mid and far periphery. The lens ring features a flange that easily conforms and adheres to the cornea and sclera when moistened in order to stabilize the lenses seated within.

Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.


Warning:

- To be used only by a licensed physician in a method consistent with other direct image ophthalmic vitrectomy lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.

Directions:

Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found with the packaging.
2. After a visual check, open the outer pouch without touching the inner packaging tray.
3. Let the tray slide onto a sterile surface. Make sure not to contaminate the tray during this process.

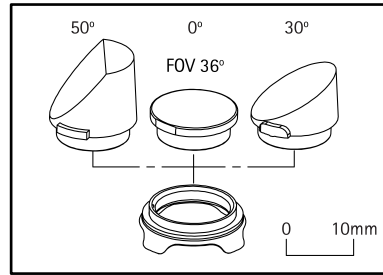Sterile operating nurse/surgtech

4. Flush cornea with ophthalmic solution.
5. Place stabilization ring on eye with foam flange contacting the eye.
6. Select the lens for the desired view: 0° lens for central posterior pole; 30° lens for posterior peripheral fundus; 50° lens for peripheral fundus.
7. While taking care not to touch the lens surface, hold the lens only by its edge and place a "tear drop" amount of ophthalmic lubricant on the curved underside of the contact lens.
8. Place lens, curved side down into the stabilization ring and hold in place with a cotton swab for several seconds to displace any air bubbles and stabilize the lens on the eye.
9. Rotate prismatic lens in stabilization flange to allow 360 posterior viewing of peripheral fundus.

Sterilization:

This item is delivered in sterile condition and is for **SINGLE USE ONLY**. After surgery, it should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

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