# HD Lens Set for HAAG-STREIT EIBOS 2 / Leica RUV800

REF 54027

en

## Instructions for Use

**Contents:**

- (1) Sterile Cover with HD Wide Field Lens
- (1) Focusing Lever
- (1) Focusing Knob

## Indications for Use:

Allows visualization of the ocular fundus, vitreous and retinal structures during vitreoretinal surgery.

## Performance Characteristics:

This lens set provides a disposable sterility cover with an integrated wide field lens, and focusing knob and lever for the HAAG-STREIT EIBOS 2 or for the LEICA RUV800 wide angle viewing systems. The HD wide field lens contained within each cover offers a high resolution view of the entire fundus and far periphery.

## Intended use:

For use with the Haag-Streit Surgical EIBOS 2 or the Leica RUV800 wide-angle viewing systems.

## Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.

## Warning:

- To be used only by a licensed physician in a method consistent with other wide-angle surgical ophthalmic lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.


## Directions: Wide-angle lenses

### Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found on the packaging.
2. After a visual check, open the outer pouch without touching the inner contents.
3. Let the contents slide onto a sterile surface. Make sure not to contaminate the contents during this process.

### Sterile operating nurse/surgtech

4. Carefully remove the blister insert containing the focusing knob and lever by lifting and pulling on the rectangular tab, taking care to not touch or make contact with the lens.
5. Push cover onto EIBOS/RUV800, grasping cover in a way to avoid contamination through the side access holes (A) for the focusing attachments (Figure 1). Push until the clip feature has passed the top of the device (B). Gently pull in the opposite direction to verify the cover has been fully secured by the clips along the top edge.
6. Remove focusing knob and lever from blister and insert them into the EIBOS/RUV800 device (Figure 2). When the device is in use, focus can be shifted from the fundus to the upper vitreous body while keeping instruments in hand by using a fingertip to move the lever. Alternatively, focusing can also be achieved by rotating the knob.

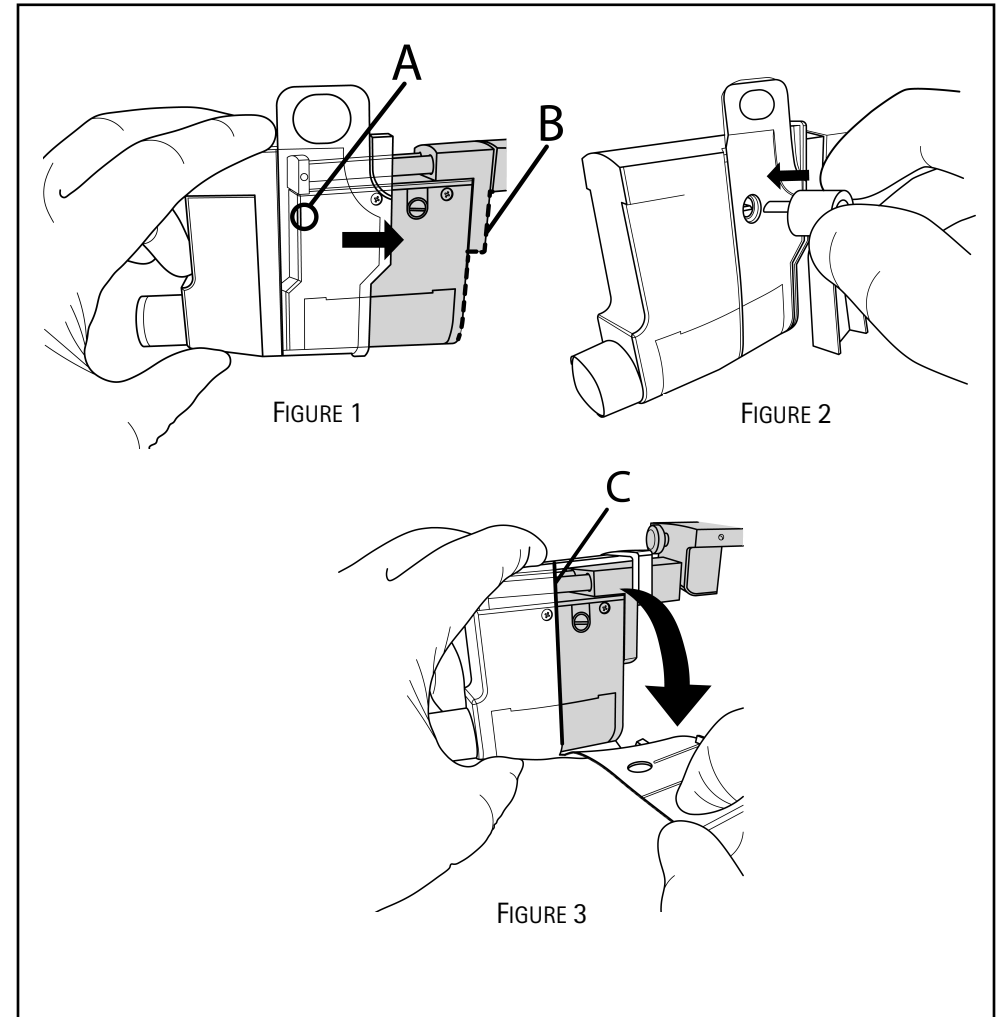7. Swing the EIBOS/RUV800 into operating position and verify smooth and unrestricted operation of the retracting safety mechanism.
8. To remove cover after use, first remove knob and lever, then firmly grasp the front and back of the device, and SLOWLY pull the tab away from the side of the device (Figure 3) until the cover separates along the tear-line (C).

## Sterilization:

These items are delivered in a sterile condition and are for **SINGLE USE ONLY**. After surgery, they should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

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