VERSA HD Disposable LenZ

REF 54216

Instructions for Use

Contents: (1) Versa HD Disposable LenZ

Indications for Use:

Allows visualization of the ocular fundus, vitreous, and retinal structures during vitreoretinal surgery. Indicated specifically for use in procedures where access to the surgical site is limited or would otherwise be blocked by the wide-angle lens.

Performance Characteristics:

This optic set provides a wide-angle view of the vitreous cavity and retina. It features a wide angle lens with a reduced profile helpful in procedures where access to the surgical site is limited or would otherwise be blocked by the wide-angle lens.

Intended use:

Intended for use with the ZEISS RESIGHT® Viewing System.

Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.

Warning:

- To be used only by a licensed physician in a method consistent with other wide-angle surgical ophthalmic lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.

Directions:

Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found on the packaging.
2. After a visual check, open the outer pouch without touching the inner packaging tray.
3. Let the tray slide onto a sterile surface. Make sure not to contaminate the tray during this process.


Sterile operating nurse/surgtech

4. Open the sterile tray and remove the lens without touching the lens surface.
5. Using your thumb and index finger, hold the lens on either side of the white arm without touching the lens surface (Figure 1). **FAILURE TO HOLD THE CLIP IN THIS MANNER MAY RESULT IN PREMATURE BREAKING OF THE CLIP.**
6. Insert the lens into the slot (Figure 2-A) and verify the lens arm is fully engaged as shown (Figure 2).
7. To remove the lens after use, grasp the lens clip as shown (Figure 1) remove, and discard.

Sterilization:

These items are delivered in a sterile condition and are for **SINGLE USE ONLY**. After surgery, they should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

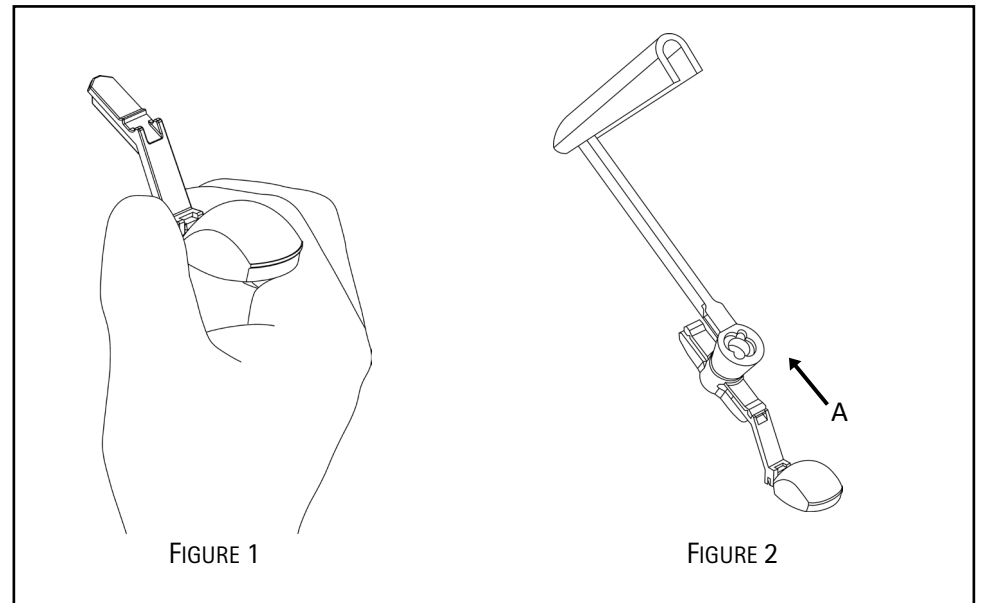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