BIOM® Versa Optic Set

REF 54211 for f=175mm

REF 54212 for f=200mm

Instructions for Use

- Contents:**
- (1) BIOM® Versa Disposable Lens
 - (1) Correction Lens
 - (2) Drive Belts
 - (1) Thumb Screw Cover (for dovetail)

Indications for Use:

Allows visualization of the ocular fundus, vitreous, and retinal structures during vitreoretinal surgery. Indicated specifically for use in procedures where access to the surgical site is limited or would otherwise be blocked by the wide-angle lens.

Performance Characteristics:

This optic set provides a wide-angle view of the vitreous cavity and retina. It features a wide angle lens with a reduced profile helpful in procedures where access to the surgical site is limited or would otherwise be blocked by the wide-angle lens.

Intended use:

Intended for use with the BIOM® Viewing System.

Objective Lens	BIOM®	BIOM® Optic Set (disposable)	REF
f=175mm	BIOM® 3 m/c, 4 m/c, 5m/c	Versa for f=175mm	54211
f=200mm	BIOM® 3 ml/cl, 4 ml/cl, 5ml/cl	Versa for f=200mm	54212

Contraindications:

There are no known contraindications, undesirable side effects, or specific patient population.

Warning:

- To be used only by a licensed physician in a method consistent with other wide-angle surgical ophthalmic lenses.
- Never use a product that appears to be damaged. Do not use if the packaging is open or damaged.
- Product is for single use only. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- In any serious incident contact manufacturer and your local competent authority.

Directions: Wide-angle lenses

Non-sterile personnel

1. Check the packaging for damage. The product must not be used if any damage is found on the packaging.
2. After a visual check, open the outer pouch without touching the inner packaging tray.
3. Let the tray slide onto a sterile surface. Make sure not to contaminate the tray during this process.

Sterile operating nurse/surgtech

4. Open the sterile tray and remove contents as needed.
5. Hold the correction lens (Figure 1-A) by its flange and place it in the lens holder (Figure 1-B), so that the mounting ring tabs on the lens fit into the notches in the lens holder.
6. Turn the correction lens clockwise, passing through the resistance of the spring clip. The correction lens is now secure and will not fall out.
7. Using your thumb and index finger, hold the wide-angle lens (Figure 1-D) on either side of the white arm without touching the lens surface (Figure 1). FAILURE TO HOLD THE CLIP IN THIS MANNER MAY RESULT IN PREMATURE BREAKING OF THE CLIP. The lens is correctly oriented if its position matches the diagram

(Figure 2). Fasten the lens by inserting the white plastic tab into the slot (Figure 1-C), passing the resistance until the small outer tabs click and lock into position.

8. To remove the lenses after use, grasp the wide-angle lens clip as shown (Figure 3) and simply break the lens clip in a firm downward motion. Remove the remaining portion of the clip and discard. Turn the correction lens counter-clockwise and carefully lift out of the lens ring holder.

Directions: Thumb screw cover

1. Grasp thumb screw cover by flat end and push over the dovetail thumb screw.
2. When tightening or loosening thumb screw, grasp cover by the round portion.
3. Remove cover after use by grasping flat end and pulling off of thumb screw.


Directions: Drive belts

Please refer to BIOM® manufacturers instructions for use.

Sterilization:

These items are delivered in a sterile condition and are for SINGLE USE ONLY. After surgery, they should be properly disposed of according to contaminated medical waste guidelines of your facility.

Contents sterile if package is not expired, opened, damaged or wet.

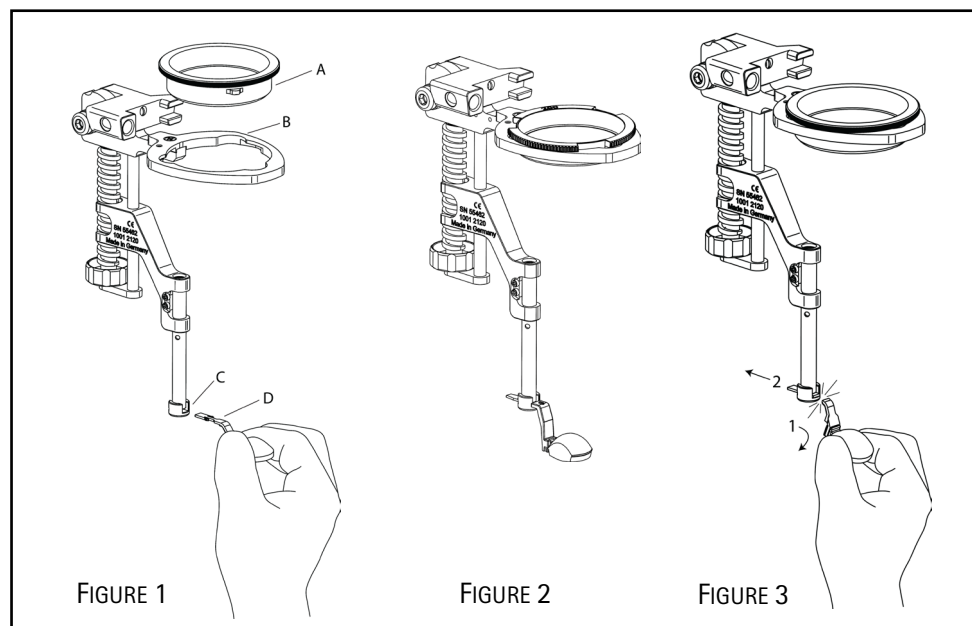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